

Preparing the

C A R T R I D G E

Vettec
ANIMAL HEALTH

BEGIN

Remove the outer cap from the cartridge




OPEN

Bump the tip of the cartridge down on a firm surface, pull up and then bend backwards to remove.

NOTE: DO NOT USE NIPPERS TO OPEN

EQUALIZE

Load the cartridge into the dispensing gun and dispense a small amount of material.

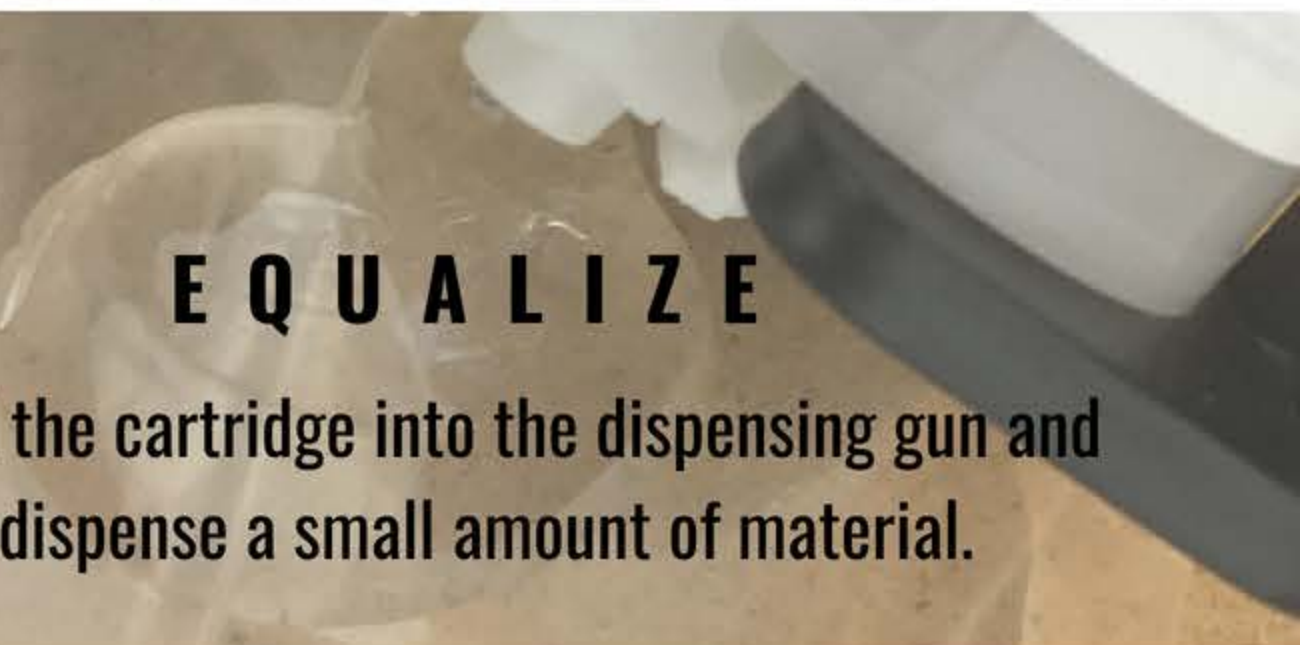

ATTACH

Place the mixing tip on the open cartridge and twist into place.



STORE

Use the cartridge cap provided to seal opened cartridges for later use.

