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SOLUBLE HEMOSTATIC GAUZE

Sterile contents

**Description:** WoundClot™ is a soluble gauze, based on cellulose structure. It is fully biocompatible and provided sterile. In contact with liquids it forms an adhesive gel which expands and adhere to the wound for bleeding control.

WOONDCLOT™ Trauma Gauze

Advanced Bleeding Control™


INSTRUCTIONS FOR USE TRAUMA DEVICE (CE)

EMERGENCY HEMOSTATIC DRESSING

For moderate and severe traumatic bleeding


Sterile contents

**Product size:** 10X10cm (4X4"), 8X20cm (3.1X8"), 8X100cm (3.1X39"), 20X30cm (8X12") . **Indications:** WoundClot™ Trauma Gauze is intended for bleeding control of traumatic wounds, including moderate to severe bleeding. **For use by trained emergency responders. Application:**

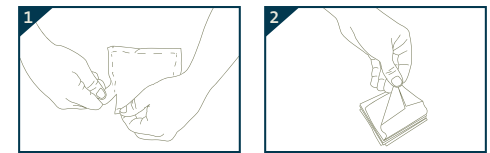1 Open the pack and take out WoundClot™ trauma gauze.

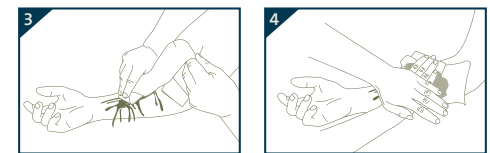2 Hold by pointed end.

3 Place the gauze gently into the wound and let the gauze soak in blood while pressing lightly to bring the gauze in contact with the bleeding area.

4 If required, place a sponge or pressure bandage gently on top of WoundClot™ trauma gauze to secure it in place until further treatment is given.

5 If bleeding persist, remove the dressing and repeat step 1-4.

**Removal:** Once hemostasis is achieved, remove by saturating the gauze with water or saline solution, then gently remove any excess gauze material away from the wound. **Cautions:** It is advised to clean the wound with water before using WoundClot™. Bleeding does not always stop immediately. If the bleeding persists pack additional gauze layer and continue to apply gentle constant pressure.

- Do not re-sterilize
- Do not use after expiration date.
- Rx Only Prescription only
- WoundClot™ gauze is a single use device. Do not reuse.
- Store in dry place
- Consult instruction for Use
- LOT Batch code
- Sterilization using Irradiation
- REF Catalog number
- Do not use if package is damaged. Sterility of contents guaranteed unless individual package is damaged or opened.
- Manufacturing Date

Manufacturer: Core Scientific Creations Ltd. 29 Nahshon Street, 4926918, Petah Tikva, Israel T: +972-3-7359555 | F: +972-3-7359544 questions@woundclot.com | www.woundclot.com

EC REP EC Rep Authorized Representative in the European Community EU Authorized Representative: CE partner4U BV, Esdoornlaan 13, 3951 DbMaarn, The Netherlands | Fax: +31 343 442 162 CE Marketing and identification number of notified body.

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Fabricante: Core Scientific Creations Ltd.