

# Elos Accurate® ScanBody22™




To ensure the ScanBody22's **reusability** and unrivalled **accuracy**, please notice some important instructions:



## Autoclave only

No ultra-sonic cleaning  
(Ultra-sonic cleaning will cause chipping of the AccuScan® surface.)

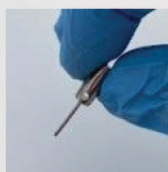

## Finger tightening only

Do not exceed 5 Ncm  
(Use the SBD-M-1 driver if needed)

## THANK YOU FOR YOUR ORDER



Congratulations on your purchase of the ScanBody22. To safeguard the product's longevity, please handle it with care and follow the recommended washing instructions.



[denthouse.se/produkter-tandlakare/scanbody-elos/](https://denthouse.se/produkter-tandlakare/scanbody-elos/)

### WASHING INSTRUCTIONS

Cycle	Minimum Time	Minimum Temperature	Type of Detergent/Water
Pre-Cleaning	2 minutes	Cold (< 45°C)	Tap water
Wash	10 minutes	Heated (50-60°C)	Neodisher MediClean Forte (0.2-1%)
Rinse	2 minutes	Cold (< 45°C)	Critical water <sup>1</sup>
Thermal Rinse	5 minutes	Heated (90°C)	Critical water <sup>1</sup>

<sup>1</sup> As per AAMI TIR34, water extensively treated usually by a multistep treatment process that may include a carbon bed, softening, DI, and RO or distillation, to ensure that the microorganisms and the inorganic and organic material are removed from the water