

detax



Premium 3D dental resins

dx 3D Guide

The background is an abstract composition of smooth, flowing, organic shapes. The primary colors are a vibrant yellow and a muted teal. The yellow shapes are more prominent, appearing as thick, curved ribbons or folds that sweep across the frame. The teal shapes are more ethereal, often appearing as soft, hazy washes or as the background behind the yellow forms. The overall effect is one of fluidity and depth, with light reflecting off the curved surfaces to create subtle gradients and highlights.

detax

Materials that matter



Welcome to the world of detax

For over 70 years, we have been dedicated to developing high-quality silicones and composites for dentistry and hearing aid acoustics. Our innovative materials empower patients to regain their quality of life and restore their smiles.

Ideas are our most important raw materials

Our passion for developing new products is our driving force – time and again, medical products from Detax set new standards in audio and dental technology.

Quality made in Ettlingen

Not only do we constantly invest in research, but we also manufacture our products ourselves in our factory on the company premises in Ettlingen. This gives us continuous control over what is most important to us in our work: its quality.

Partnership to go

Medical products from Detax are valued in over 100 countries around the world. To ensure safe distribution, Detax works with selected partner companies in the target countries.

The best thing about us is the we

A respectful attitude towards our business partners and our staff is important to us. Friendly appreciation determines the way we treat each other and people outside the company.

Detax is growing and our teams are also expanding, which is why we welcome every application. From initial contact and onboarding to update meetings and further training: We accompany and support every employee in all phases of their working life.

3D resins by detax

denture/C&B



Material type							
Application		Removable denture bases, total prosthesis	Removable denture bases, total prosthesis	Removable partial dentures, flexible	Permanent crowns, denture teeth, Long-term temporary bridges	Temporary crowns & bridges	Individual functional try-ins
Color		Pink-transparent, pink	Pink-transparent, pink	Pink-transparent, clear	A1, A2, A3, A3.5, B1, B3, C2, D3, BL	A1, A2, A3	A1, A2, A3
Medical Device Class	MDR	Ila	Ila	Ila	Ila	Ila	Ila
	FDA	II	II	II	II	II	I
	NMPA	–	–	–	–	–	–
	HC	II	II	II	III	II	I

model



Material type							
Application		Master & working models, situation models, control models	Master & working models, situation models, control models	Working models for thermoforming and aligner technology	Self-isolating working models	Thermoforming models	Gingival masks for dental models
Color		Caramel, grey, light grey, sand	Caramel, grey, sand	Caramel, light tan	Stone	Light blue	Gingiva
Medical Device Class	MDR	TEC resin	TEC resin	TEC resin	TEC resin	TEC resin	TEC resin
	FDA	TEC resin	TEC resin	TEC resin	TEC resin	TEC resin	TEC resin
	NMPA	I	–	–	–	I	I
	HC	TEC resin	TEC resin	TEC resin	TEC resin	TEC resin	TEC resin

aligner/splint/ surgical guide



Material type						
Application		Direct printing of aligners and retainers	Functional splints, retainers, mouth-guards, night-guards, bleaching trays	Flexible splints, retainers, mouth-guards, night-guards, bleaching trays	Hard splints	Autoclavable surgical guides, orthodontic base components
Color		Clear-transparent	Clear-transparent	Clear-transparent	Clear-transparent	Clear-transparent
Medical Device Class	MDR	Ila	Ila	Ila	Ila	Ila
	FDA	in process	II	II	I	I
	NMPA	–	–	–	TEC resin	TEC resin
	HC	II	II	II	II	II

others



Material type				
Application		Individual impression trays, functional trays, base plates	Orthodontic bracket transfer trays, bleaching trays	Casting technique, burns without residue
Color		Green	Transparent	Red-transparent
Medical Device Class	MDR	I	I	TEC resin
	FDA	I	I	TEC resin
	NMPA	MED resin	–	–
	HC	I	I	TEC resin

MDR Medical Device Regulation EU
 FDA Food and Drug Administration USA
 NMPA National Medical Products Administration China
 HC Health Canada

3D Freeprint® Material

denture/C&B



3D Freeprint® Material

denture

Light-curing formulation for 3D printing of denture bases and total prosthesis.

**Colors:**
pink-transparent,
pink

**Wavelength:**
385 nm

**Medical Product:**
Class IIa

REF	Product	Unit
02060 / 02040 / 03518	Freeprint® denture pink-transparent	500 g / 1000 g / 5 kg
04092 / 03298	Freeprint® denture pink	1000 g / 5 kg


Parameters	Standard	
Flexural strength	DIN EN ISO 20795-1 ¹⁾	> 100 MPa
Flexural modulus	DIN EN ISO 20795-1 ¹⁾	> 2500 MPa
Water absorption	DIN EN ISO 20795-1 ¹⁾	< 32 µg / mm³
Solubility	DIN EN ISO 20795-1 ¹⁾	< 1.6 µg / mm³
Hardness	–	> 83 Shore D
Biocompatibility	DIN EN ISO 10993-1 ²⁾	fulfilled

¹⁾ Dentistry: Denture resins (in keeping with the standard at room temperature)
²⁾ Biological assessment of medical devices – Part 1: Assessment and testing in the context of a risk management system







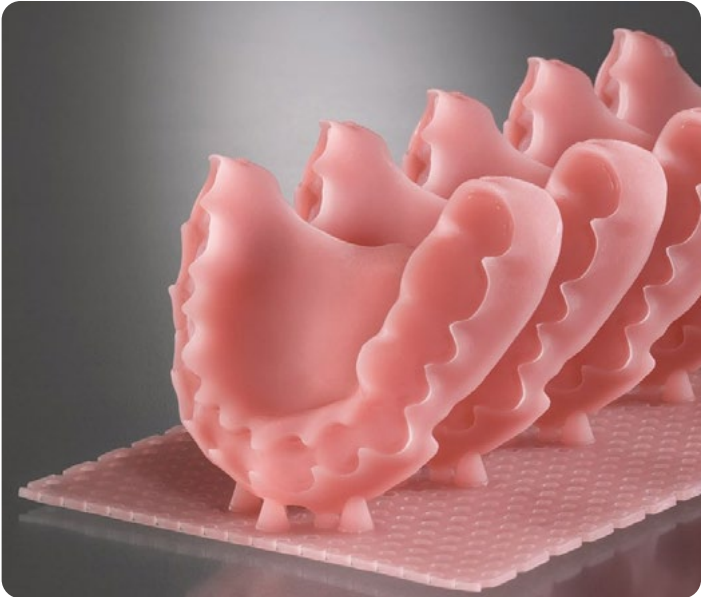

3D Freeprint® Material

denture impact

Light-curing formulation for 3D printing of impact resistant denture bases.



Colors:
pink-transparent,
pink



Wavelength:
385 nm



Medical Product:
Class IIa

REF	Product	Unit
04436	Freeprint® denture impact pink-transparent	1000 g
04437	Freeprint® denture impact pink	1000 g

Parameters	Standard	
Flexural strength	DIN EN ISO 20795-1 ¹⁾	> 80 MPa
Flexural modulus	DIN EN ISO 20795-1 ¹⁾	> 2100 MPa
Water sorption	DIN EN ISO 20795-1 ¹⁾	< 32 µg / mm ³
Solubility	DIN EN ISO 20795-1 ¹⁾	< 1.6 µg / mm ³
Hardness	–	> 82 Shore D
Biocompatibility	DIN EN ISO 10993-1 ²⁾	fulfilled

¹⁾ Dentistry: Denture resins (in keeping with the standard at room temperature)

²⁾ Biological assessment of medical devices – Part 1: Assessment and testing in the context of a risk management system



3D Freeprint® Material

denture flex

Light-curing formulation for 3D printing of flexible partial denture bases.


Colors:
 pink-transparent,
 clear


Wavelength:
 385 nm


Medical Product:
 Class IIa

REF	Product	Unit
04625	Freeprint® denture flex pink-transparent	1000 g
04626	Freeprint® denture flex clear	1000 g

Parameters	Standard	
Elongation	DIN EN ISO 527-1 ¹⁾	> 20 %
Tensile Strength	DIN EN ISO 527-1 ¹⁾	> 45 MPa
Water sorption	DIN EN ISO 20795-1 ²⁾	< 32 µg / mm ³
Solubility	DIN EN ISO 20795-1 ²⁾	< 1.6 µg / mm ³
Hardness	–	> 78 Shore D
Biocompatibility	DIN EN ISO 10993-1 ³⁾	fulfilled

¹⁾ Resins: Determination of tensile strength (in keeping with the standard at room temperature)
²⁾ Dentistry: Denture resins (in keeping with the standard at room temperature)
³⁾ Biological assessment of medical devices – Part 1: Assessment and testing in the context of a risk management system














3D Freeprint® Material

crown

Light-curing formulation for 3D printing of permanent single crowns, denture teeth and long-term temporary bridges.

**Colors:**

A1, A2, A3, A3.5,
B1, B3, C2, D3, BL

**Wavelength:**

385 nm

**Medical Product:**

Class IIa

REF	Product	Unit
02372 / 02376	Freeprint® crown A1	500 g / 1000 g
02378 / 02415	Freeprint® crown A2	500 g / 1000 g
02417 / 02446	Freeprint® crown A3	500 g / 1000 g
04709 / 04708	Freeprint® crown A3.5	500 g / 1000 g
02481 / 02519	Freeprint® crown B1	500 g / 1000 g
02645 / 02758	Freeprint® crown B3	500 g / 1000 g
02766 / 02782	Freeprint® crown C2	500 g / 1000 g
02783 / 02825	Freeprint® crown D3	500 g / 1000 g
02845 / 02884	Freeprint® crown BL	500 g / 1000 g

Parameters	Standard	
Flexural strength	DIN EN ISO 10477 ¹⁾	> 100 MPa
Flexural modulus	DIN EN ISO 10477 ¹⁾	> 2800 MPa
Water absorption	DIN EN ISO 10477 ¹⁾	< 40 µg / mm ³
Solubility	DIN EN ISO 10477 ¹⁾	< 7.5 µg / mm ³
Hardness	–	> 50 Barcol
Biocompatibility	DIN EN ISO 10993-1 ²⁾	fulfilled

¹⁾ Crown and veneering resins (in keeping with the standard at room temperature)

²⁾ Biological assessment of medical devices – Part 1: Assessment and testing in the context of a risk management system



3D Freeprint® Material

temp

Light-curing formulation for 3D printing of temporary crowns & bridges and anterior and posterior tooth restorations.


Colors:
A1, A2, A3


Wavelength:
385 nm


Medical Product:
Class IIa

REF	Product	Unit
04058 / 04062	Freeprint® temp A1	500 g / 1000 g
04059 / 04063	Freeprint® temp A2	500 g / 1000 g
04060 / 04064	Freeprint® temp A3	500 g / 1000 g

Parameters	Standard	
Flexural strength	DIN EN ISO 10477 ¹⁾	> 100 MPa
Flexural modulus	DIN EN ISO 10477 ¹⁾	> 2300 MPa
Water absorption	DIN EN ISO 10477 ¹⁾	< 40 µg / mm³
Solubility	DIN EN ISO 10477 ¹⁾	< 7.5 µg / mm³
Hardness	–	> 40 Barcol
Biocompatibility	DIN EN ISO 10993-1 ²⁾	fulfilled

¹⁾ Crown and veneering resins (in keeping with the standard at room temperature)
²⁾ Biological assessment of medical devices – Part 1: Assessment and testing in the context of a risk management system




36 months



detax MDR CERTIFIED


FDA 510K CLEARED


HEALTH CANADA CLASS II LICENSE


Premium MMA free THF-MA free Formula


CE 0483

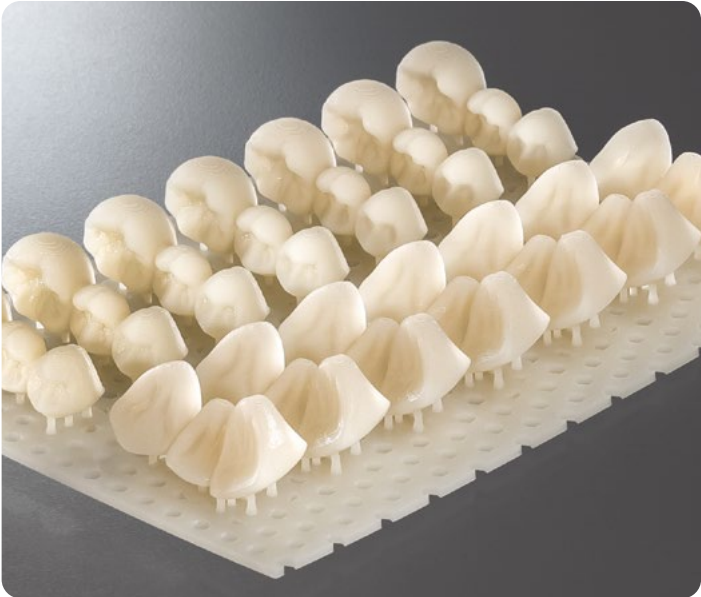

3D Freeprint® Material

tryin

Light-curing formulation for 3D printing of individual functional try-ins of digitally manufactured denture bases.



Colors:
A1, A2, A3



Wavelength:
385 nm



Medical Product:
Class IIa

REF	Product	Unit
04426	Freeprint® tryin A1	1000 g
04427	Freeprint® tryin A2	1000 g
04428	Freeprint® tryin A3	1000 g

Parameters	Standard	
Flexural strength	DIN EN ISO 20795-1 ¹⁾	> 100 MPa
Flexural modulus	DIN EN ISO 20795-1 ¹⁾	> 2200 MPa
Hardness	–	> 85 Shore D
Biocompatibility	DIN EN ISO 10993-1 ²⁾	fulfilled

¹⁾ Dentistry: Denture resins (in keeping with the standard at room temperature)

²⁾ Biological assessment of medical devices – Part 1: Assessment and testing in the context of a risk management system






detax

Materials that matter

3D Freeprint® Material

aligner/splint/surgical guide

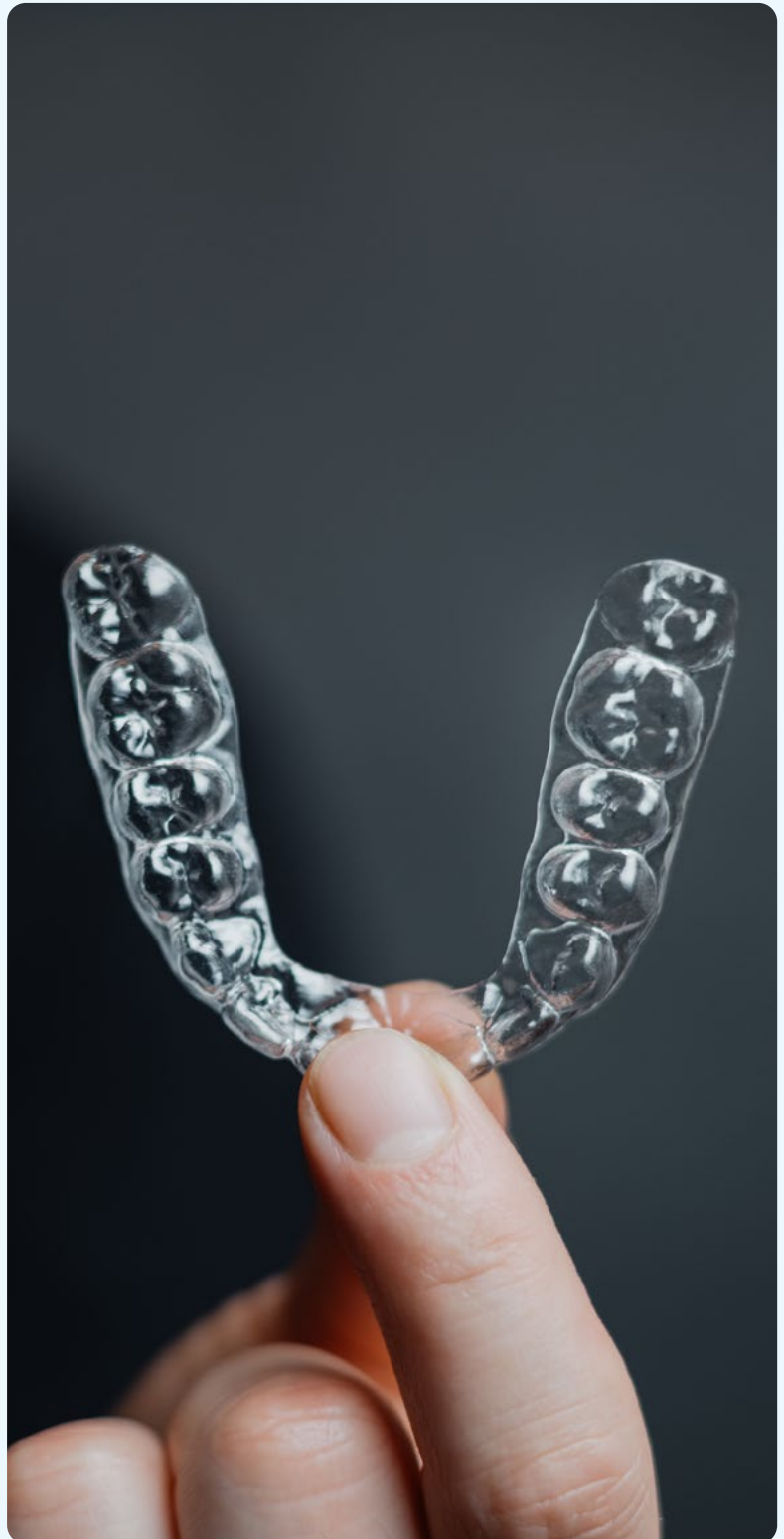

3D Med Material

dx direct aligner

Light curing formulation for direct printing of aligners and retainers.

- Outstanding biocompatibility
- Optimal balance of strength & flexibility
- Streamlined workflow
- No discoloration


Color:
clear-transparent


Wavelength:
385 nm


Medical Product:
Class IIa

REF	Product	Unit
04707	dx direct aligner	1000 g
04706	dx direct aligner	5 kg

Parameters	Standard	
Flexural strength	DIN EN ISO 20795-2 ¹⁾	> 40 MPa
Flexural modulus	DIN EN ISO 20795-2 ¹⁾	> 1000 MPa
Tensile strength	DIN EN ISO 527-1 ²⁾	> 55 MPa
Elongation	DIN EN ISO 527-1 ²⁾	> 25 %
Hardness	-	> 75 Shore D
Glass transition temperature	DIN EN ISO 6721-11 ³⁾	> 86 °C
Water absorption	DIN EN ISO 20795-2 ¹⁾	< 32 µg/mm ³
Solubility	DIN EN ISO 20795-2 ¹⁾	< 5 µg/mm ³
Biocompatibility	DIN EN ISO 10993-1 ⁴⁾	fulfilled

¹⁾ Dentistry: Orthodontic resins (in keeping with the standard at room temperature)
²⁾ Resins: Determination of tensile strength (in keeping with the standard at room temperature)
³⁾ Resins: Determination of dynamic mechanical properties - Glass transition temperature
⁴⁾ Biological assessment of medical devices – Part 1: Assessment and testing in the context of a risk management system













3D Freeprint® Material

splintmaster taff & flex

Light-curing formulation for 3D printing of flexible splints, retainers, mouthguards, nightguards and bleaching trays. In two levels of flexibility: taff – for functional splints, flex – for flexible splints.



Color:
clear-transparent



Wavelength:
385 nm



Medical Product:
Class IIa

REF	Product	Unit
04433 / 04581	Freeprint® splintmaster taff	1000 g / 5 kg
04432 / 04580	Freeprint® splintmaster flex	1000 g / 5 kg

Parameters	Standard	taff / flex
Tensile strength	DIN EN ISO 527-1 ¹⁾	> 40 MPa / > 25 MPa
Tensile elongation	DIN EN ISO 527-1 ¹⁾	> 20 % / > 50 %
Tear propagation resistance	DIN EN ISO 34-1 ²⁾	> 140 N/mm / > 110 N/mm
Hardness	–	> 75 Shore D / > 65 Shore D
Water absorption	DIN EN ISO 20795-2 ³⁾	< 32 µg / mm³ / < 32 µg / mm³
Solubility	DIN EN ISO 20795-2 ³⁾	< 5 µg / mm³ / < 5 µg / mm³
Biocompatibility	DIN EN ISO 10993-1 ⁴⁾	fulfilled / fulfilled

¹⁾ Resins: Determination of tensile strength (in keeping with the standard at room temperature)

²⁾ Thermoplastic elastomers: Determination of tear propagation resistance (in keeping with the standard at room temperature)

³⁾ Dentistry: Orthodontic resins (in keeping with the standard at room temperature)

⁴⁾ Biological assessment of medical devices – Part 1: Assessment and testing in the context of a risk management system



3D Freeprint® Material

splint 2.0

Light-curing formulation for 3D printing of hard splints.

 **Color:**
clear-transparent

 **Wavelength:**
385 nm

 **Medical Product:**
Class IIa

REF	Product	Unit
02080 / 02076	Freeprint® splint 2.0	500 g / 1000 g


Parameters	Standard	
Flexural strength	DIN EN ISO 20795-2 ¹⁾	> 80 MPa
Flexural modulus	DIN EN ISO 20795-2 ¹⁾	> 2000 MPa
Water absorption	DIN EN ISO 20795-2 ¹⁾	< 32 µg / mm³
Solubility	DIN EN ISO 20795-2 ¹⁾	< 5 µg / mm³
Hardness	–	> 80 Shore D
Biocompatibility	DIN EN ISO 10993-1 ²⁾	fulfilled

¹⁾ Dentistry: Orthodontic resins (in keeping with the standard at room temperature)
²⁾ Biological assessment of medical devices – Part 1: Assessment and testing in the context of a risk management system







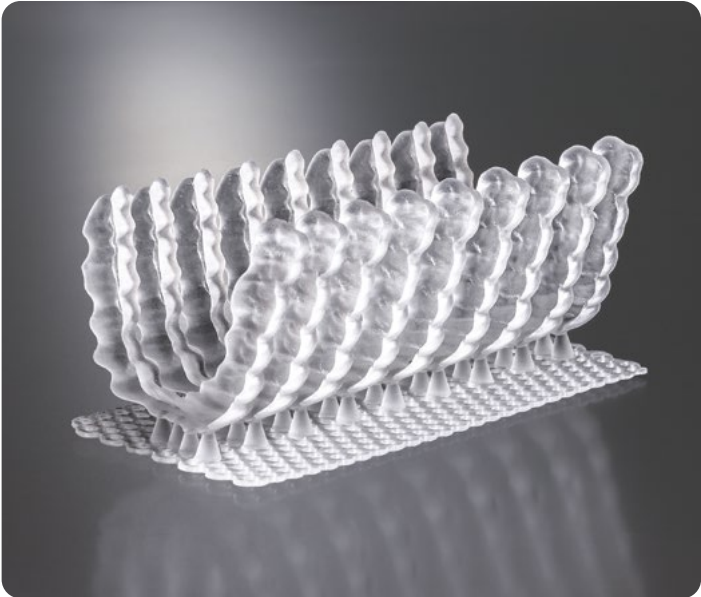

3D Freeprint® Material

ortho

Light-curing formulation for 3D printing of autoclavable base parts for orthodontic appliances, surgical guides and X-ray templates.



Color:
clear-transparent



Wavelength:
385 / 405 nm



Medical Product:
Class IIa

REF	Product	Unit
03989 / 04323	Freeprint® ortho 385	1000 g / 5 kg
03988	Freeprint® ortho 405	1000 g

Parameters	Standard	
Flexural strength	DIN EN ISO 20795-2 ¹⁾	> 75 MPa
Flexural modulus	DIN EN ISO 20795-2 ¹⁾	> 1650 MPa
Water absorption	DIN EN ISO 20795-2 ¹⁾	< 32 µg / mm ³
Solubility	DIN EN ISO 20795-2 ¹⁾	< 5 µg / mm ³
Hardness	–	> 82 Shore D
Biocompatibility	DIN EN ISO 10993-1 ²⁾	fulfilled

¹⁾ Dentistry: Orthodontic resins (in keeping with the standard at room temperature)

²⁾ Biological assessment of medical devices – Part 1: Assessment and testing in the context of a risk management system




The background is an abstract composition of smooth, flowing, organic shapes in shades of yellow, green, and teal. The shapes overlap and curve, creating a sense of movement and depth. The lighting is soft, with subtle gradients and highlights that give the shapes a three-dimensional appearance.

detax

Materials that matter

3D Freeprint® Material

model

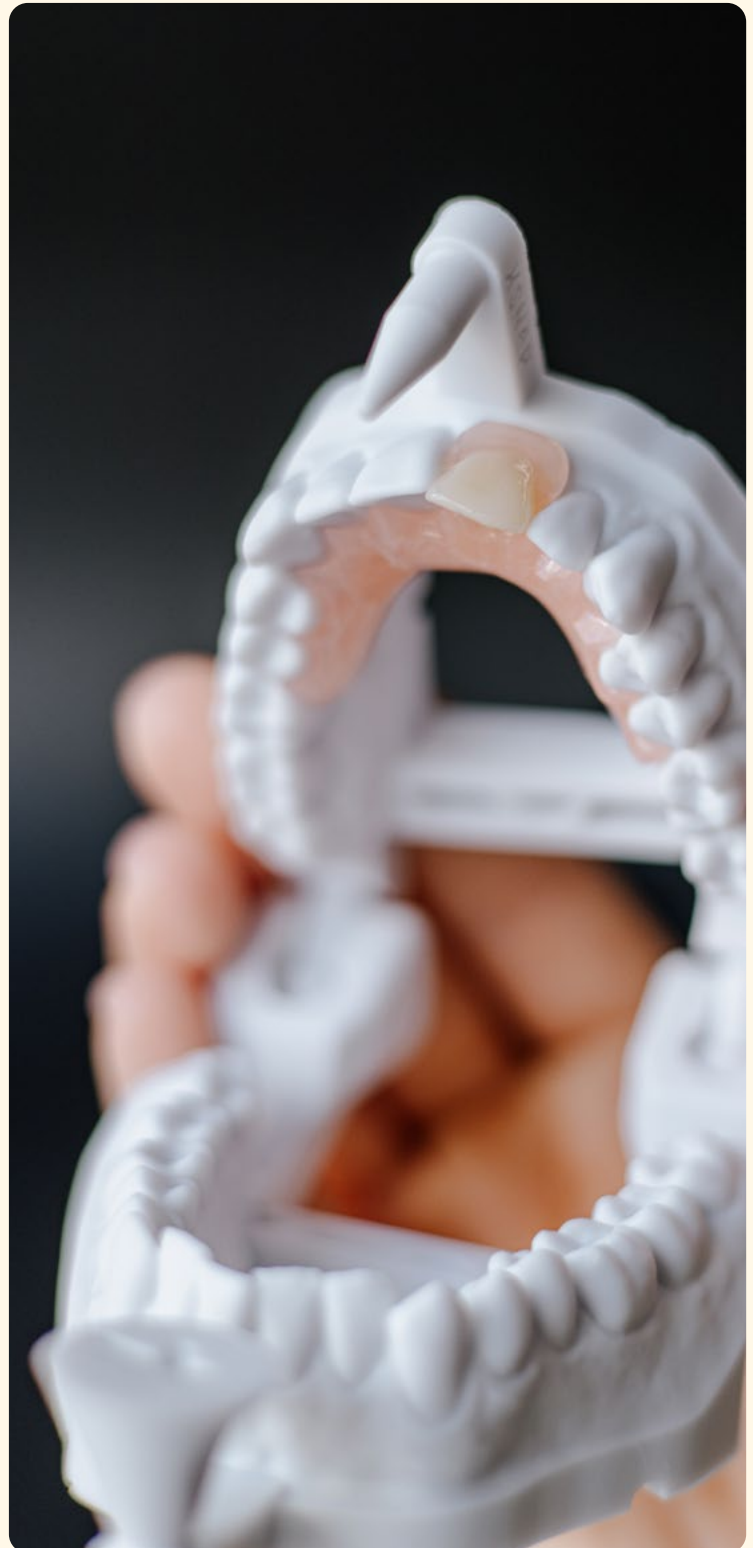

3D Freeprint® Material

model 2.0

Light-curing formulation for 3D printing of master and working models, situation models, control models.

**Colors:**
caramel, light grey, grey, sand, white

**Wavelength:**
380 – 405 nm

**Technical product**

REF	Product	Unit
02850 / 04015	Freeprint® model 2.0 caramel	1000 g / 5 kg
02099 / 04107	Freeprint® model 2.0 light grey	1000 g / 5 kg
02177 / 04106	Freeprint® model 2.0 grey	1000 g / 5 kg
02128 / 04117	Freeprint® model 2.0 sand	1000 g / 5 kg
02148 / 04118	Freeprint® model 2.0 white*	1000 g / 5 kg

* not THF-MA free

Parameters	Standard	
Flexural strength	DIN EN ISO 178 ¹⁾	> 80 MPa
Flexural modulus	DIN EN ISO 178 ¹⁾	> 1700 MPa
Hardness	–	> 84 Shore D

¹⁾ Resins: Determination of flexural strength (in keeping with the standard at room temperature)







3D Freeprint® Material

model pro

Light-curing formulation for 3D printing of master and working models, situation models, control models.

**Colors:**
caramel, grey, light grey, sand

**Wavelength:**
380 – 405 nm

**Technical Product**

REF	Product	Unit
04440 / 02585	Freeprint® model pro caramel	1000 g / 5 kg
04438 / 02574	Freeprint® model pro grey	1000 g / 5 kg
02546 / 02558	Freeprint® model pro light grey	1000 g / 5 kg
04439 / 02579	Freeprint® model pro sand	1000 g / 5 kg

Parameters	Standard	
Flexural strength	DIN EN ISO 178 ¹⁾	> 90 MPa
Flexural modulus	DIN EN ISO 178 ¹⁾	> 2000 MPa
Hardness	–	> 82 Shore D

¹⁾ Resins: Determination of flexural strength (in keeping with the standard at room temperature)









3D Tec Material

dx model basic

Light-curing formulation for 3D printing of working models, especially for thermoforming and aligner technology.

- High temperature resistance
- Good surface hardness
- Short post-processing
- Suitable for hollow models



Colors:
caramel, light tan



Wavelength:
380 – 405 nm



**Technical
Product**



REF	Product	Unit
04734	dx model basic caramel	1000 g*
04735	dx model basic caramel	5 kg*
04733	dx model basic caramel	200 kg*
04737	dx model basic light tan	1000 g*
04738	dx model basic light tan	5 kg*
04736	dx model basic light tan	200 kg*

* Minimum order quantity 500 kg

Parameters	Standard	
Flexural strength	DIN EN ISO 178 ¹⁾	> 75 MPa
Flexural modulus	DIN EN ISO 178 ¹⁾	> 2000 MPa
Hardness	–	> 80 Shore D

¹⁾ Resins: Determination of flexural strength (in keeping with the standard at room temperature)



3D Tec Material

dx iso model pro

Light-curing formulation for 3D printing of self-isolating working models.

- Excellent release properties against (meth)acrylate-based polymers
- Ready to use – no additional release agents required
- Low viscosity for easy and fast cleaning



Color:
stone



Wavelength:
380 – 405 nm



Technical Product

REF	Product	Unit
04758	dx iso model pro stone	1000 g
04759	dx iso model pro stone	5 kg
04705	dx iso model pro stone	200 kg

Parameters	Standard	
Flexural strength	DIN EN ISO 178 ¹⁾	> 45 MPa
Flexural modulus	DIN EN ISO 178 ¹⁾	> 1300 MPa
Hardness	–	> 75 Shore D

¹⁾ Resins: Determination of flexural strength (in keeping with the standard at room temperature)



3D Freeprint® Material

model T

Light-curing formulation for 3D printing of thermoforming models.

 **Color:**
light blue

 **Wavelength:**
380 – 405 nm

 **Technical product**

REF	Product	Unit
02332 / 04322	Freeprint® model T	1000 g / 5 kg

Parameters	Standard	
Working temperature for thermoforming sheets	–	≤ 195 °C
Flexural strength	DIN EN ISO 178 ¹⁾	> 80 MPa
Flexural modulus	DIN EN ISO 178 ¹⁾	> 1700 MPa
Hardness	–	> 83 Shore D

¹⁾ Resins: Determination of flexural strength (in keeping with the standard at room temperature)



 36 months

 eco

 TEC

 Premium
MMA free
Formula



3D Freeprint® Material

gingiva

Light-curing formulation for 3D printing of flexible gingival masks for dental models.

 **Color:**
gingiva

 **Wavelength:**
380 – 405 nm

 **Technical product**

REF	Product	Unit
02820 / 02843	Freeprint® gingiva	500 g / 1000 g

Parameters	Standard	
Tensile strength	DIN EN ISO 527-1 ¹⁾	> 3 MPa
Tensile elongation	DIN EN ISO 527-1 ¹⁾	> 90 %
Hardness	–	> 70 Shore A

¹⁾ Resins: Determination of tensile strength (in keeping with the standard at room temperature)




 36 months

 TEC

 Premium
MMA free
THF-MA free
Formula

 BPA
free

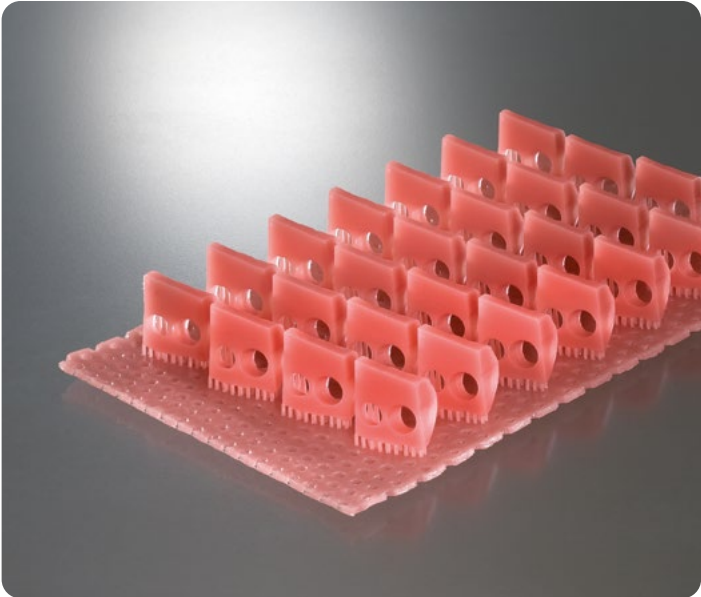



detax

Materials that matter

3D Freeprint® Material

tray/ibt/cast



3D Freeprint® Material

tray 2.0

Light-curing formulation for 3D printing of individual impression and functional trays, base plates.



Color:
green



Wavelength:
380 – 405 nm



Medical Product:
Class I

REF	Product	Unit
02505	Freeprint® tray 2.0	1000 g
04624	Freeprint® tray 2.0	5 kg

Parameters	Standard	
Flexural strength	DIN EN ISO 178 ¹⁾	> 90 MPa
Flexural modulus	DIN EN ISO 178 ¹⁾	> 1900 MPa
Hardness	–	> 84 Shore D
Biocompatibility	DIN EN ISO 10993-1 ²⁾	fulfilled

¹⁾ Resins: Determination of flexural strength (in keeping with the standard at room temperature)
²⁾ Biological assessment of medical devices – Part 1: Assessment and testing in the context of a risk management system













dx

29

3D Freeprint® Material

ibt

Light-curing formulation for 3D printing of flexible orthodontic bracket transfer trays and bleaching trays.



Color:
transparent



Wavelength:
385 nm



Medical Product:
Class I


REF	Product	Unit
04249	Freeprint® ibt	1000 g

Parameters	Standard	
Tensile strength	DIN EN ISO 527-1 ¹⁾	> 8 MPa
Tensile elongation	DIN EN ISO 527-1 ¹⁾	> 60 %
Tear propagation resistance	DIN EN ISO 34-1 ²⁾	> 35 N/mm
Hardness	–	> 90 Shore A
Biocompatibility	DIN EN ISO 10993-1 ³⁾	fulfilled

¹⁾ Resins: Determination of tensile strength (in keeping with the standard at room temperature)

²⁾ Thermoplastic elastomers: Determination of tear propagation resistance (in keeping with the standard at room temperature)

³⁾ Biological assessment of medical devices – Part 1: Assessment and testing in the context of a risk management system

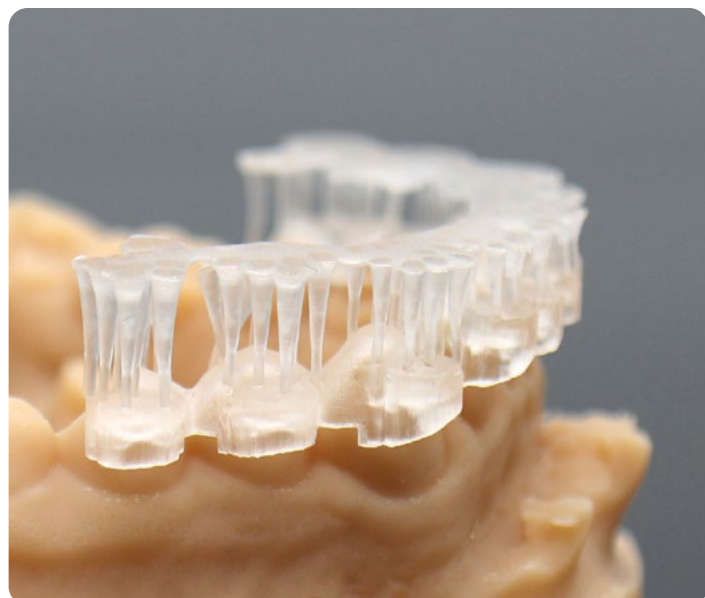

3D Freeprint® Material

cast 2.0

Light-curing formulation for 3D printing of high-precision casting objects.


Color:
red-transparent


Wavelength:
380 – 405 nm


Technical product

REF	Product	Unit
02548 / 02632	Freeprint® cast 2.0	500 g / 1000 g

Parameters	Standard	
Flexural strength	DIN EN ISO 178 ¹⁾	> 70 MPa
Flexural modulus	DIN EN ISO 178 ¹⁾	> 1700 MPa
Bakeout temperature	–	1 h @ 800 °C
Combustion residue	–	< 0.1%

¹⁾ Resins: Determination of flexural strength (in keeping with the standard at room temperature)




36 months


TEC


Premium
MMA free
THF-MA free
Formula



dx validation printer matrix (385 nm)

	denture/C&B					aligner/splint/surgical guide					model					tray/lbt/cast			
	denture	denture impact	denture flex	crown	temp	tylin	aligner	splint	surgical guide		model 2.0	model pro	dx model basic	dx model pro	model T	gingiva	tray 2.0	lbt	cast 2.0
ASIGA	Max	●	●	●	●	●	●	●	●		●	●	●	●	●		●		
	Max2	●	●	●	●	●	●	●	●		●	●	●	●	●		●		
	Ultra	●	●	●	●	●	●	●	●		●	●	●	●	●		●		
	Pico2	●			●	●			●		●				●		●		
	PRO2	●			●	●			●		●				●		●		
	PRO 4K	●	●	●	●	●	●	●	●		●	●	●	●	●		●		
rapidshope	ONE		●	●	●	●	●	●	●									●	
	Pro20	●	●	●	●	●	●	●	●		●	●			●		●	●	●
	Pro30	●	●	●	●	●	●	●	●		●	●			●		●	●	●
	D10/D20 Series	●	●	●	●	●	●	●	●		●	●			●		●	●	●
	D30/D40 Series	●	●	●	●	●	●	●	●		●	●			●		●	●	●
	D50+ Series	●	●	●	●	●	●	●	●						●	●	●	●	●
straumann	D70/D90 Series				●										●				
	D100 Series													●					
	P-Series	●	●	●	●	●	●	●	●		●	●			●		●	●	●
	Qualification	●	●	●	●	●	●	●	●		●	●			●		●	●	●

Date: 06/2026

● validated ● in process

dx validation printer matrix (385 nm)

	denture/C&B	aligner/splint/surgical guide	model	tray/lbt/cast
ivoclar				
PrograPrint PRS				
MICROLAY [®] 3D PRINTING				
Versus				
Gildewell				
Fastprint.Li				
LuxCreo				
iLux Pro Dental				
MICRAFT				
Prime/Hyper Series				
Ultra Series				
Alpha				
Hy-Clean				
A2D				
A2D HD				
A3D				
Chairside				
W2P				
SoFlex				
Qualification	● validated	● in process		

dx validation printer matrix (405 nm)

	denture/C&B										aligner/splint/surgical guide										model										tray/tbt/cast																			
	formilabs																																																	
Form 4B																																																		
RAYSHAPE®																																																		
Edge E2																																																		
Edge Mini																																																		
MICROLAY® DENTAL																																																		
Eve Pro																																																		
Aldite																																																		
CPD-100																																																		
phrozen																																																		
Sonic 4K 2022																																																		
SHINING 3D DENTAL																																																		
AccuFab L4D																																																		
AccuFab CEL																																																		
AccuFab D1s																																																		
MTC																																																		
Mecatronmore																																																		
Qualification																																																		
	● validated										● in process										● validated										● in process										Date: 06/2026									

dx validation curing matrix

dx validation curing matrix


	denture/C&B						aligner/splint/surgical guide						model						tray/lbt/cast			
TC																						
BB-Cure																						
BB-Cure N																						
phrozen																						
Phrozen Cure																						
ivoclar																						
PrograPrint Cure																						
formlabs																						
Form Cure (2nd Generation)																						
Form Cure																						
Heraeus																						
PCU Pro																						
PCU 3.0																						
Qore																						
PCU LED N2																						
SHINING 3D																						
FabCure																						
Qualification																						
	● validated	● in process																				

Date: 06/2026

Date: 06/2026

dx validation curing matrix

[illegible]

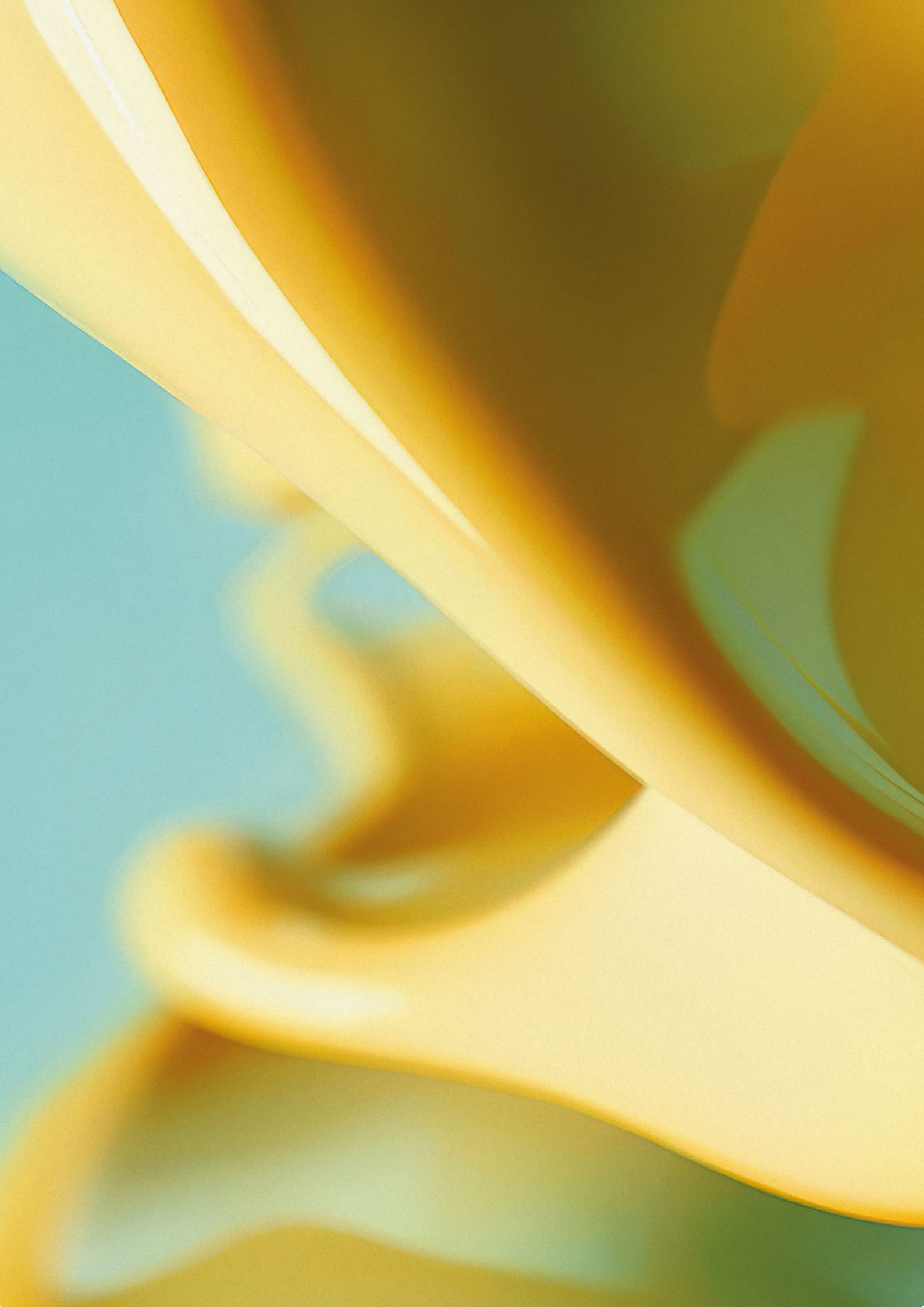

detax

Discover our best 3D resins



FDA ✓

MDR ✓

Health Canada ✓



detax

detax GmbH
Carl-Zeiss-Str. 4 • 76275 Ettlingen/Germany
T +49 7243 510 0 • F +49 7243 510 100
post@detax.com • detax.com