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 **D20108** **RFX+CARE®**
17.5x9.5cm
1pcs

     **MD** **Medical device** 
PAP

 **RFX+CARE Manufacturing Co., Ltd.**
7 Lanjiang Road, Yeucheng District,
Shaoxing, Zhejiang, China 312000

EC **REP** **RFX+CARE International A/S**
Bakkegaardsvej 408, 3050
Humlebæk, Denmark

GB
Face mask
Type IIR approved in accordance with EN 14683: 2014
Intended use:
Protection against inhalation of airborne particles such as transmitted bacteria, vira as well as dust and pollen.
Instruction for use:
1. Wash your hands before wearing. Gently stretch the upper and lower sides to expand the face mask. The blue side with the nasal bridge clamp (black if it is B/W mask), should face outwards when you are wearing the mask.
2. Make sure that the nasal bridge clamp is facing upward. Make sure that the mask is tightly fitted to the face also placing the ear string firmly around ears.
3. Gently press the nasal bridge clamp around the nose to ensure a tight fit and adhesion.
4. Spread and unfold until it covers the chin completely, leaving no gaps between the mask cheeks and chin.
Warning:
Single use. Do not use if the mask is dirty or broken. Keep away from children.
Non sterile.
Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

D20108-IFU
2020-03

5 706636 565986