



## New Pool Lift Chair

New Pool Lift Chair is the ideal solution for swimmers who have mobility problems but can still sit on a chair on their own.

These are the main advantages of the new Pool Lift Chair:

- usable both in combination with our mobile pool elevator and with our ceiling elevator.
- Design in sturdy plastic with an ergonomic sitting position
- Very easy to use and safe with a standard integrated waist belt
- The lift chair fits in every (shower) wheelchair
- Suitable for both children and adults
- More lifting height (9,70 à 12,50 cm)

More technical details can be found in the product sheet and on the website:

<https://www.handimove.com/products/pool-lift-chair>

The new Pool Lift Chair is now available with reference #3620.



## Comparing the old seat sling and the new rigid Pool Lift Chair.

We first did a study on what could be improved on the existing seat sling and the following were the main things that came out of that study :

- The old seat sling folds together in the water (meaning a little bit more tricky to get back in, definitely independently)
- The old seat sling does not sink (not enough weight)
- Too much adjustability (the person could be put in a position that was too much reclined)
- Depending on the user: pressure on the legs due to the material being used

## We have developed the new Pool Lift Chair with following improvements:

- The new Pool Lift Chair is rigid (doesn't fold together ), yet provides some flexibility for comfort thanks to the material being used (PVC)
- The use of PVC also gives it a higher density than water, therefore it will not float (the specific locations of the drain holes help here as well)
- The new Pool Lift Chair is slightly curved to follow the spine of the person and accommodates (dimension wise) to P95 of the population and can be used with most common wheel- and shower chairs.

