

GeniX ROM - Knee orthosis

ART.NO 70646

Postoperative knee orthosis, used after injury or surgery to the knee joint, where controlled range of motion or total immobilization is needed. The orthosis is developed to enable quick and easy fitting after injury/surgery. GeniX ROM has a very low weight and is comfortable to wear.

With the telescopic mechanism, the length of the orthosis can be adjusted, from approximately 45 cm to 64 cm in 0.8 cm increments. The splints and cuffs can be bent to be individually adapted to each patient. All cuffs are removable. The inside of the orthosis is provided with soft padding, as well as anti-slip silicone coating that ensures that the orthosis remains in place. Cuttable straps with quick buckles simplify fitting and putting on and taking off for the patient.

The range of motion can be set for extension from -10° to 90°, flexion from 0° to 120°, both in intervals of 10°. The ROM setting is secure and does not change on its own during use. Quick-release button fix the ROM in a range from -10° to 30°.

Indications: Postoperative and posttraumatic immobilization/stabilization of the knee joint.

Contraindications: Not intended for people with weight >110 kg.

Material: Aluminum, polyoxymethylene, polyamide, polyurethane, viscose, elastane, silicone. Latex free.

Washing instructions: Textile padding: Wash at 40°C, for best durability hand wash is recommended, dry flat. The stays and joints: Wash with a cloth and warm water/ mild disinfectant.

Size	Art.no
GeniX ROM Universal	706460050

Accessories	Art.no
GeniX ROM Straps	706460054
GeniX ROM Padding	706460055



The aluminum cuffs can be shaped to fit the leg.



Quick release buckle on the straps.

