

# Heel wedges for Walkers

ART.NO 80215, 80224, 80225

**Indications:** Achilles tendon rupture. As an insert in a walker boot to increase plantar flexion in ankle joint when needed.

**Contraindications:** None known.

**Function:** All wedges are intended to be placed in a walker to increase plantar flexion as per the treatment plan for Achilles rupture. Double-sided tape between the wedges keeps them in place.



80215 - Cirrus heel wedge, left/right.

## Cirrus, heel wedge Left/Right - 80215:

5 wedges with plateau, anatomically shaped for right/left foot. To adjust plantar flexion, select the number of wedges in relation to foot size and desired plantar flexion. Available in two variants right/left, both in universal size.

**Material:** EVA, PA. Latex free.

Product:	Size	Art.no Universal
Cirrus, Left heel wedge	Universal	802151000
Cirrus, Right heel wedge	Universal	802152000



80224 - Heel wedge 10°, without plateau, bilateral.

## Walker Wedge Kit 10° - 80224:

3 wedges, without plateau. The wedges are placed on top of each other to achieve the desired plantar flexion. 3 wedges = 30° plantar flexion, 2 wedges = 20° plantar flexion, 1 wedge = 10° plantar flexion. Available in two sizes, bilateral.

**Material:** EVA. Latex free.

Product:	Art.no Size 1	Art.no Size 2
Walker Wedge Kit 10°	802240040	802240060



80225 - Stratus heel wedge, bilateral.

## Stratus, Universal heel wedge - 80225:

4 wedges and a sole, with plateau. To adjust plantar flexion, select the number of wedges in relation to foot size and desired plantar flexion. Universal size, bilateral.

**Material:** Sole: SBR, PES, PA. Wedges: PE. Latex free.

Product:	Size	Art.no Universal
Stratus, Universal heel wedge	Universal	802250000

MD Class I CE

**Washing Instructions:** Clean with a soft damp cloth use mild detergent.