# ComfortSling

ARM SLING

Art no 20111-09-50

Universal size

12 pcs.

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**EN** Arm sling for support of the shoulder/arm in the event of acute injuries.

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**EN** Place arm in the sling pocket as shown in the illustration. Pull strap back over your shoulder until buttonhole comfortable aligns with the orange button. Place button through buttonhole securely. Strap may be secured on either side of the button. If necessary the strap can be shortened to desired length.

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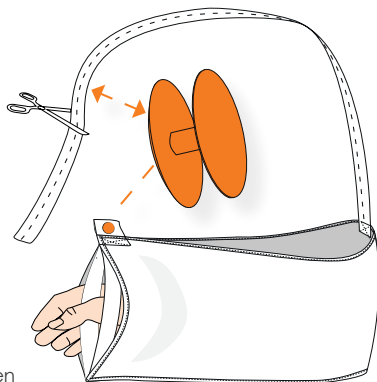
**EN** For support of the shoulder/arm in the event of acute injuries.

**Contraindications:** For patients with sinus carotid sensitivity avoid pressure from the orthosis in the carotid sinus area.

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**EN** If a stinging sensation, rash or other changes in skin color occur, discontinue use and contact your doctor or other medically trained person. Only medically trained persons are to prescribe and recommend the use of the product. For best results, follow instructions for use. Use in combination with other medical products should be supervised by qualified medical practitioner. Serious incidents shall be reported to NordiCare and the competent authority.

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For further questions, contact your doctor, health practitioner or medical equivalent - or NordiCare: +46 42-35 22 20, [info@nordicare.se](mailto:info@nordicare.se).