
Duo

art no 20114

SHOULDER IMMOBILIZER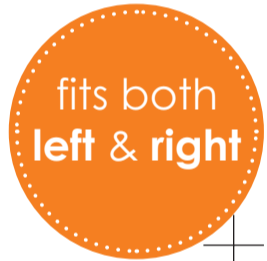

Size	Length
XL	48-55 cm
L	43-49 cm
M	40-44 cm
S	35-41 cm

DISPOSAL OF THE PRODUCT
Do not dispose the product in the nature. Dispose of the product in accordance with the waste laws of your municipality and/or country. The product does not present a risk of contamination.

This packaging can be recycled as paper and the bag as plastic.

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EN Shoulder immobilizer for immobilization and fixation of the shoulder and upper arm. To be used in case of acute injuries, trauma or postoperative. Four sizes, universal right/left.

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- EN** 1. Place the arm in the sling so that the elbow rests against the rear edge. Close the top edge using (C).
2. Thread the shoulder strap (A) through the D-ring at (C). Make sure that the fastening (C) is located right above the wrist (so that the hand is supported).
3. Attach (B) to the front of the arm pocket (A). Pull (B) behind the back.
4. Attach (B) at the wrist of (A).
5. Done. If necessary the thumb strap (D) can be used.

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Material:

Nylon 40%, PU Foam 25%, Polyester 20%, Cotton 15%.
Latex free.

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EN If a stinging sensation, rash or other changes in skin color occur, discontinue use and contact your doctor or other medically trained person. Only medically trained persons are to prescribe and recommend the use of the product. For best results, follow instructions for use. Periodically check the condition of the Velcro to ensure they are free from grease and damage. Use in combination with other medical products should be supervised by qualified medical practitioner. Serious incidents shall be reported to NordiCare and the competent authority.

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EN Machine wash at 40°C (106°F). Seal the Velcros. Use washing bag. Do not bleach. Do not dry clean. Do not iron. Do not tumble dry.

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