
Easy rehab training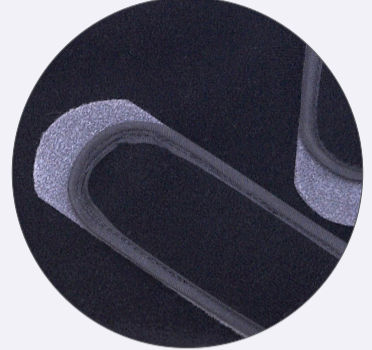


Extra velcro plates for secure closure

## DuoTrain

art no 20119

### SHOULDER IMMOBILIZER

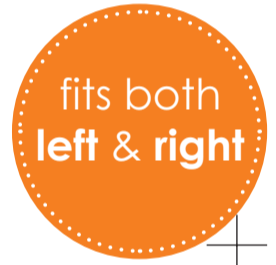2025-10-01

DISPOSAL OF THE PRODUCT  
Do not dispose the product in the nature.  
Dispose of the product in accordance with the waste laws of your municipality and/or country. The product does not present a risk of contamination.

Size	Length
XL	48-55 cm
L	43-49 cm
M	40-44 cm
S	35-41 cm

This packaging can be recycled as paper and the bag as plastic.

MD Class I CE

Phone +46 42 35 22 20,  
Fax +46 42 35 22 21  
info@nordicare.se,  
www.nordicare.se

Solrosvägen 1,  
SE-263 62 Viken  
Sweden

NordiCare®

**EN** Used to relieve/immobilize the shoulder/arm during post-op/acute injuries/trauma when a shoulder immobilizer that enables repeated opening/closing capability is required, for example when exercising. **IMPORTÄNT!** Always use the gray velcro plates according to the user instructions.

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#### EN Important before use:

- Gray velcro plates should always be used.
- The sling is mounted to fit the right arm, for use on left arm: turn/move waistband (E) and velcro plates (B).

- Place the arm in the sling so that the elbow rests against the rear edge. Close the top edge using (C + D).
- Thread the shoulder strap (A) through the D-ring at (D). Make sure that the fastening (D) is located right above the wrist (so that the hand is supported).
- Fasten waist strap (E) on the velcro plate (B) at the wrist, if necessary, move (B) to the desired location.
- Adjust the length of the waist strap at the back.

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EN For immobilization/fixation of shoulder/arm. In case of trauma, emergency or postoperative care.





**Contraindications:** For patients with sinus carotid sensitivity avoid pressure from the orthosis in the carotid sinus area.

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### Material:

Nylon 42%, PU-foam 23%, Polyester 21%, Cotton 14%. Latex free.

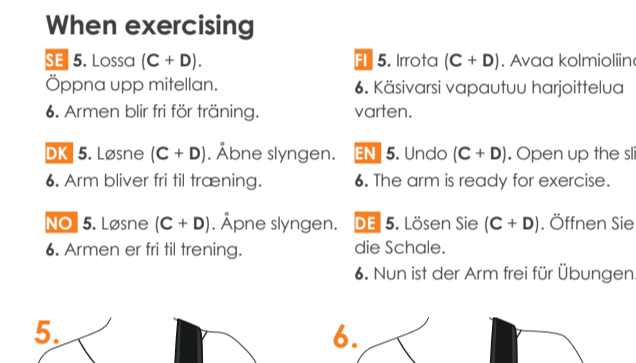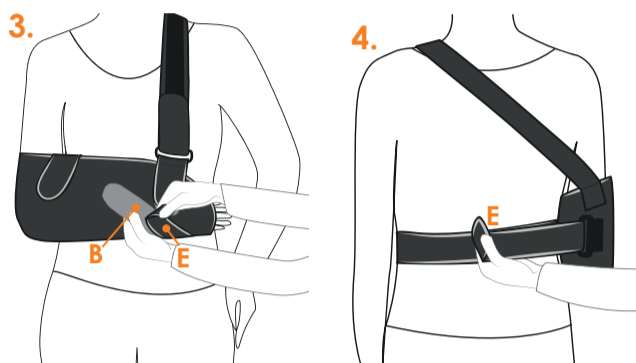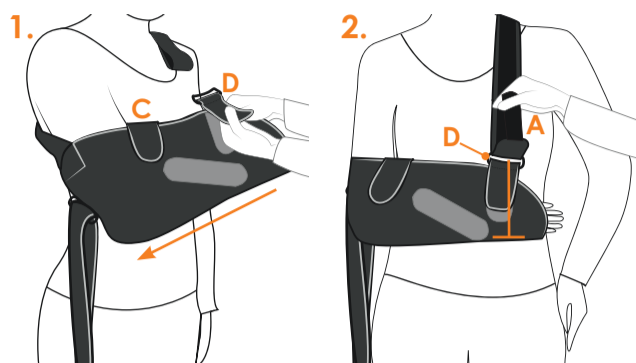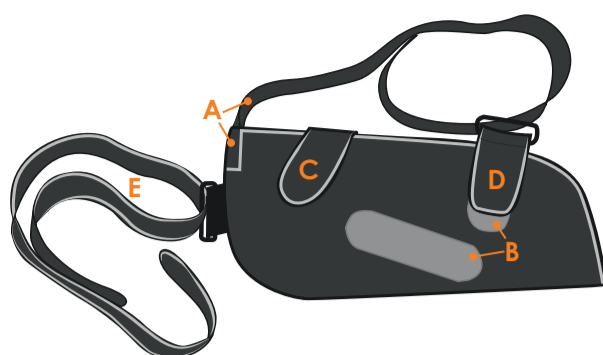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EN If a stinging sensation, rash or other changes in skin color occur, discontinue use and contact your doctor or other medically trained person. Only medically trained persons are to prescribe and recommend the use of the product. For best results, follow instructions for use. Use in combination with other medical products should be supervised by qualified medical practitioner. Serious incidents shall be reported to NordiCare and the competent authority.

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### Care and washing instructions

EN Periodically check the condition of the Velcro to ensure they are free from grease and damage. Remove DuoTrain before taking a bath/shower. Seal the Velcros before washing, machine wash at 40°C (104°F), can be washed at 60°C (140°F) if necessary. Use of a laundry bag is recommended.

For more information please consult our product catalogue or visit [www.nordicare.se](http://www.nordicare.se). For further questions, contact your doctor, health practitioner or medical equivalent - or NordiCare: +46 42-35 22 20, [info@nordicare.se](mailto:info@nordicare.se).