## DuoQuick

art no 20124

### SHOULDER IMMOBILIZER

This packaging can be recycled as paper and the bag as plastic.

Phone +46 42 35 22 20,  
Fax +46 42 35 22 21  
info@nordicare.se,  
www.nordicare.se

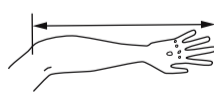
Solrosvägen 1,  
SE-263 62 Viken  
Sweden

NordiCare®

DISPOSAL OF THE PRODUCT  
Do not dispose the product in the nature.  
Dispose of the product in accordance with the waste laws of your municipality and/or country. The product does not present a risk of contamination.

Size	Length
XL	48-55 cm
L	43-49 cm
M	39-44 cm
S	35-40 cm
XS	30-35 cm

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EN Shoulder immobilizer fixates the shoulder and arm, provides support and relief in case of acute injury, trauma or postoperative. Fits both left and right arm. Available in size XS-XL.

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**EN** For immobilization/fixation of shoulder/arm. In case of trauma, emergency or postoperative care.

**Contraindications:** For patients with sinus carotid sensitivity avoid pressure from the orthosis in the carotid sinus area.

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### Material:

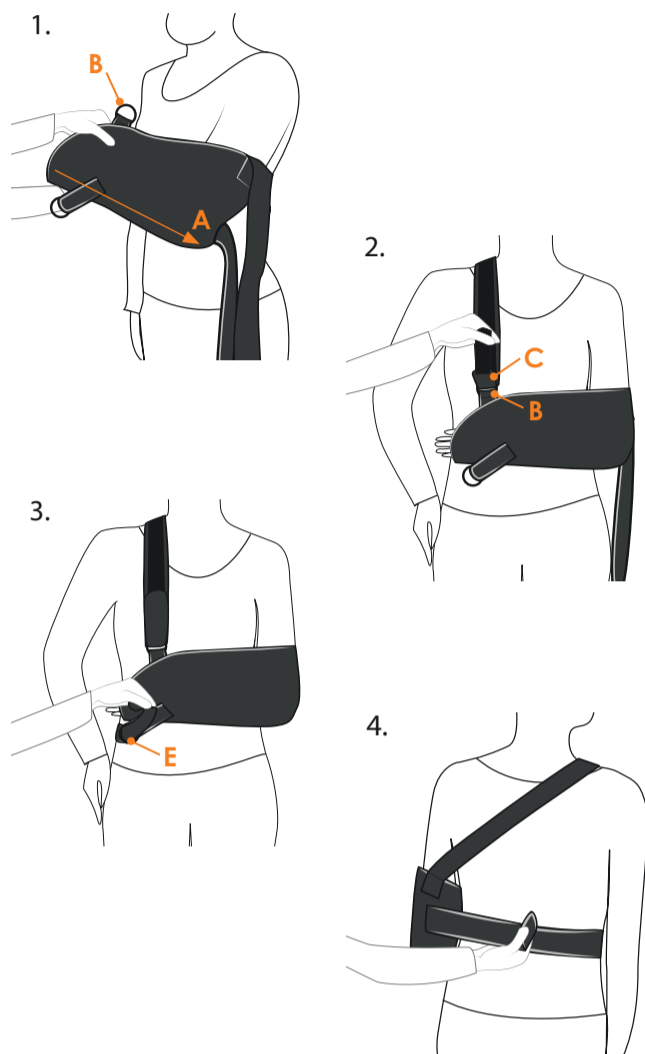
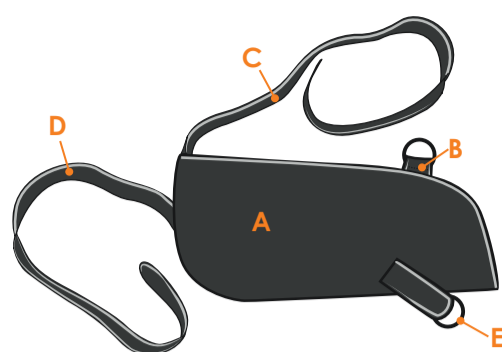
77% Polyester, 10% Cotton, 9% Polyurethane, 4% Nylon Latex free.

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**EN** If a stinging sensation, rash or other changes in skin color occur, discontinue use and contact your doctor or other medically trained person. Only medically trained persons are to prescribe and recommend the use of the product. For best results, follow instructions for use. Use in combination with other medical products should be supervised by qualified medical practitioner. Serious incidents shall be reported to NordiCare and the competent authority.

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**EN** Machine wash at 40°C (106°F). Seal the Velcros. Use washing bag. Do not bleach. Do not dry clean. Do not iron. Do not tumble dry.

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**NordiCare Ortopedi & Rehab AB**