INSTRUCTIONS FOR USE

HumerusComfort - HUMERAL FRACTURE BRACE

art no 20220

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EN If the responsible medical practitioner allows the orthosis to be removed during e.g. shower or washing.

1. First loosen the fixation band around the torso (A) completely.

2. Loosen the two straps (B1) and (B2) around the arm carefully. Carefully remove the orthosis. To minimize pain and strain around the fracture, lean to the side of the fracture while removing the orthosis. Let the arm hang vertically down towards the floor. Note how the straps are applied to be able to reseat them in equal manner when putting the orthosis on again.

3. When putting on, help of one person is needed. Put on the orthosis from the bottom, up over the arm and shoulder.

4. Thread the fixation band from the back under the armpit and through the D-ring (C). Fasten the velcro so that the orthosis fits correctly over the shoulder.

5. First tighten the upper strap (B1) and then the lower strap (B2) around the arm by simultaneously pulling each strap end and securing with the Velcro closure. Tighten properly so that the orthosis is firmly in place but be careful to avoid the risk of stasis.

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art no 20220

Material Composition

Polyester 44%, Polyamide 36%, Elastane 10%, Cotton 2%, Polypropylene 7%, Polyethylene 1%. Latex free.

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EN Machine wash at 40°C (104°F). Remove the shells and seal the Velcros. Use washing bag. Do not bleach. Do not dry clean. Do not iron. Do not tumble dry.

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EN Orthosis for diaphyseal humerus fracture.

Contraindications: Massive soft tissue damage, poor compliance alternative. Unacceptable fracture position.

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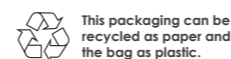
For more information please consult our product catalogue or visit www.nordicare.se. For further questions, contact your doctor, health practitioner or medical equivalent - or NordiCare: +46 42-35 22 20, info@nordicare.se.

NordiCare
2025-09-16

NordiCare Ortopedi & Rehab AB
Solrosvägen 1, SE-263 62 Viken, Sweden
Phone +46 42 35 22 20, Fax +46 42 35 22 21
info@nordicare.se, www.nordicare.se

DISPOSAL OF THE PRODUCT
Do not dispose the product in the nature. Dispose of the product in accordance with the waste laws of your municipality and/or country. The product does not present a risk of contamination.

Registered community design by OHIM: 006260816-0001

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EN

- A:** Outer orthosis part with D-rings (A1, A2)
B: Inner orthosis part with two straps (B1, B2)
C: Fastener with D-ring and velcro Y-gap
D: Fixation band with velcro Y-gap
E: Outer shell
F: Inner shell
G: Tiles

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EN 1. Start by preparing outer orthosis part (A) for adaptation to the right or left arm. The fixation band (D) must be attached to the back side and the D-ring C to the chest side of (A). Thread the two straps (B1) and (B2) through the D-rings (A1) and (A2) and detach the inner orthosis part (B).

2. Pull on the outer orthosis part (A) from below and up over the shoulder with both hands.

3. Thread the fixation band (D) under the armpit and through the D-ring (C) and fasten with Velcro. Ensure that the orthosis fits correctly over the shoulder by tightening the band around the torso. Adjust the length of the fixation band (D) by removing the velcro y-gap and cut the strap to the desired length.

4. Check that the outer shell (E) has the sufficient length based on the location of the fracture and the length of the upper arm. If a longer or shorter orthosis is needed, HumerusComfort or HumerusComfort Extend can be chosen instead.

5. Adjust the D-rings on the outer orthosis part (A) taking into account the location of the fracture.

6. Place the inner orthosis part (B) on the inside of the upper arm, as far up towards the armpit as possible. The inner orthosis part (B) must be placed so that any overlap is on the outside of the outer orthosis part (A), with the wider part of the inner orthosis part upwards. If the inner shell (F) needs to be adapted to a shorter or longer length, the orthosis can be supplemented with an Axilla Kit HC in a different size.

7. The tiles (G) in the side pockets can be removed if they are not needed for the orthosis to reach around the patient's upper arm. If there is a large overlap between (A) and (B), remove the tiles (G).

8. Thread the lower straps (B1, B2) through the D-rings (A1, A2) and tighten to fixate the fracture in the orthosis. First fasten the upper strap (B1), then the lower strap (B2) around the arm by simultaneously pulling each strap end and securing with the velcro closure. Tighten properly so that the orthosis is firmly in place, but be careful to avoid the risk of stasis.

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