INSTRUCTIONS FOR USE

CubitiComfort - HUMERAL FRACTURE BRACE

art no 20222

User instruction.

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- EN 1.** If allowed by the doctor/fitter, the orthosis can be removed e.g. when showering. First, completely loosen the shoulder strap **A**. Loosen the two Velcro straps **B** around the upper arm a little.
- 2.** Carefully remove the orthosis. Make a note of how all the straps are tightened so that you can tighten them back up again properly. If the orthosis cannot be taken off for bathing/showering, it is important that it is protected from moisture.
- 3.** Get someone to help you put the orthosis back on again. Pull the orthosis up the arm until it is sitting properly on the shoulder. Pull the shoulder strap around and beneath the opposite armpit, pull it through the D-ring and attach it using the Velcro fastener.
- 4.** Next tighten first the upper then the lower strap on the upper arm. Then, do the same for the lower arm.
- NB!** The brace must sit firmly and comfortably in place, it is important that it is not too tight in order to avoid stasis. If you experience throbbing in your arm, change in skin colour or skin irritation, loosen the straps a little and contact the fitter.

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Material Composition

36% Polyamide, 44% Polyester, 10% Elastane, 2% Cotton, 7% Polypropylene, 1% Polyethylene. Latex free. Hinge: 100% Aluminum.

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- EN** Orthosis for diaphyseal humerus fracture.
Contraindications: Massive soft tissue damage, poor compliance alternative. Unacceptable fracture position.

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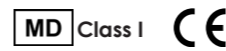
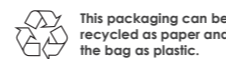
For more information please consult our product catalogue or visit www.nordicare.se. For further questions, contact your doctor, health practitioner or medical equivalent - or NordiCare: +46 42-35 22 20, info@nordicare.se.

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EN If a stinging sensation, rash or other changes in skin color occur, discontinue use and contact your doctor or other medically trained person. Only medically trained persons are to prescribe and recommend the use of the product. For best results, follow instructions for use. Periodically check the condition of the Velcro to ensure they are free from grease and damage. Use in combination with other medical products should be supervised by qualified medical practitioner. Serious incidents shall be reported to NordiCare and the competent authority.

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Registered community design by OHIM: 001951500-0001

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EN

- A: Outer orthosis part
- B: D-rings
- C: Inner orthosis part with two straps
- D: Fastener with D-ring and velcro Y-gap
- E: Fixation band with velcro Y-gap

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EN 1. Start by preparing the brace for use on the right or left arm. The fixation band **E** should be attached on the back side and the fastener **D** on the chest side. Detach and remove the two straps around the arm section, and the inner brace section **C**. Open up the forearm brace section.

2. Slide on the arm section from the bottom and slide the brace up over the shoulder using both hands. Then thread the fixation band **E** through the fastener **D** and secure it with the Velcro. Adapt the fixation band **E** length by removing the loose velcro Y-gap and cutting the strap to the required length.

3. Make sure the brace sits stable over the shoulder by tensioning the strap around the chest.

4. Make sure the hinge is centered over the elbow. By changing the length of the rods the position of the hinge can be adjusted. Loosen the screws without undoing them completely, adjust and then re-tighten the screws. Adjust the length on the forearm brace section similarly.

ROM: The product is supplied with two screws for adjusting the mobility of the elbow hinge. Set the desired range of motion.

5. Make sure that the integrated plates on the upper arm and on the forearm

are in desired length. If the plates needs to be adjusted, remove and cut the plates to the desired length. To reduce the risk of pressure in the crook of the elbow during movement, the corners can be rounded/cut off.

6. Fasten the D-rings on the outer brace section **A**, paying attention to where the fracture is located. Position the inner brace section **C** on the inside of the upper arm wrapped over the outside of the outer brace section **A** with the wider part of the panel facing up.

7. First tighten the upper strap around the arm by simultaneously pulling both strap ends and fastening with the Velcro. Then tighten the lower strap in the same way. Pull tight!

8. Tighten the strap on the forearm brace section.

NOTE: The brace must sit firmly and comfortably in place, it is important that it is not too tight in order to avoid stasis.

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
EN Circumference around the upper arm and forearm proximal. The total length of the orthosis can be adjusted to desired fit.

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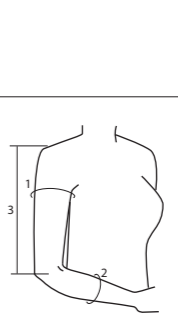Accessory - Axilla Kit HC - art no 20219

To be ordered separately, if necessary for individual adaptation or when needed to replace the inner part.

Size	Length	Art.no. Axilla Kit HC:	
S	12 cm	20219-00-40	
M	14 cm	20219-00-50	
L	16 cm	20219-00-60	
XL	18 cm	20219-00-70	

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EN X. Slim arm: Circular compression is achieved without files.


Y. Larger arm: Circular compression is achieved by means of files, which are placed in pockets of the outer orthosis part **A**.

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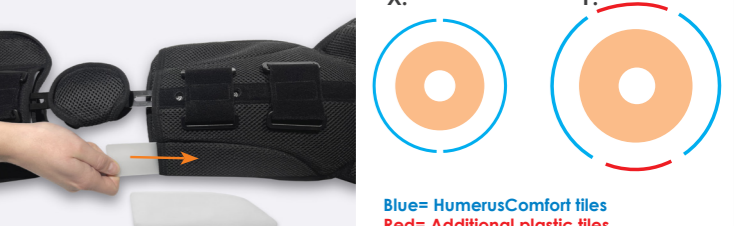Blue= HumerusComfort tiles
Red= Additional plastic tiles

Additional plastic tiles can be ordered separately.