50101 - Hög

50100 - Låg

50100 - Låg

50101 - Hög

Atle

art no 50100, 50101

BACK SUPPORT

LEVEL OF SUPPORT - LIGHT

DISPOSAL OF THE PRODUCT
Do not dispose the product in the nature. Dispose of the product in accordance with the waste laws of your municipality and/or country. The product does not present a risk of contamination.

This packaging can be recycled as paper and the bag as plastic.

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EN Soft back support in breathable, elastic material that provides flexible fit for different body shapes. Elastic straps and moldable stays for adjustable support. Suitable for low back pain, especially suitable for vertebral compression caused by osteoporosis.

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EN **Atle Låg:** Lumbar/lumbosacral back pain.

Atle Hög: Lumbal/lumbothoracal back pain.

Contraindications: Unstable fracture of the back. Sciatica pain of unclear genesis.

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EN To determine the size, measure hip circumference in cm. Atle is available in two different heights.

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Material Composition

83% Polyester, 8% Nylon, 7% Spandex, 2% Natural rubber latex*.

*Latex is covered does not come in contact with skin.

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EN Machine wash at 40°C (104°F). Remove the stays. Seal the Velcros. Use washing bag. Do not bleach. Do not dry clean. Do not iron. Do not tumble dry.

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EN If a stinging sensation, rash or other changes in skin color occur, discontinue use and contact your doctor or other medically trained person. Only medically trained persons are to prescribe and recommend the use of the product. For best results, follow instructions for use. Periodically check the condition of the Velcro to ensure they are free from grease and damage. Use in combination with other medical products should be supervised by qualified medical practitioner. Serious incidents shall be reported to NordiCare and the competent authority.

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For more information please consult our product catalogue or visit www.nordicare.se. For further questions, contact your doctor, health practitioner or medical equivalent - or NordiCare: +46 42-35 22 20, info@nordicare.se.