Information

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EN Product fitting, prescribing and recommendations of use should be done by or under the supervision of a qualified medical practitioner. A qualified medical practitioner should also supervise when using this product in combination with other medical products. Follow instruction for best result. Periodically check the condition of the Velcro to ensure they are free from grease and damage. In case of pain, chafing or colour changes in the skin, remove the orthosis and contact the fitter. Serious incidents shall be reported to NordiCare and the competent authority.

Material

49% Polyester, 25% Natural rubber latex, 14% Silicone, 12% Polyamid.

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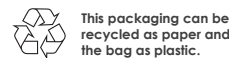
EN Machine wash, 40°. Close the Velcro fastenings. Use a washing net. Do not bleach, do not dry clean. Do not iron or tumble dry.

For more information please consult our product catalogue or visit www.nordicare.se. For further questions, contact your doctor, health practitioner or medical equivalent - or NordiCare: +46 42-35 22 20, info@nordicare.se

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EN 1. Place the belt over the pubis so that the cross-stitch and NordiCare label are in the center front. The surface with silicone edges should be placed towards the body.

2. Place the straps behind your back with both hands. Switch the straps around behind your back.

3. Tighten the belt until desired support and fasten it with the Velcro straps at the front.

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
EN Counteract loosening of the joints/symphysis during or after pregnancy. Also suitable as stable sacroiliac support for non-pregnant patients. **Contraindications:** None known.

Measurement

Size	Hip circumference
S/M	75-110 cm
L/XL	110-145 cm

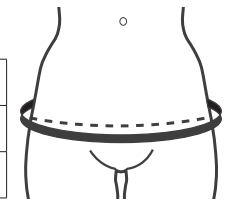Tip if you want to shorten the belt:

If the belt is too long, you can cut at the marked seam along the edges of the strap. To prevent the material from fraying zig-zag the cut edge or carefully burn it with a flame to melt the material.