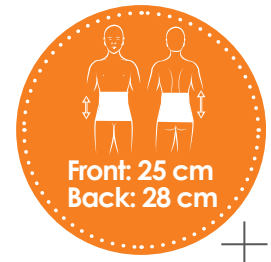
AtlePlus

art no 50105

BACK SUPPORT

LEVEL OF SUPPORT - LIGHT

| Size | Hip Circumference |
|------|-------------------|
| 4XL | 145-155 cm |
| 3XL | 135-145 cm |
| 2XL | 125-135 cm |
| XL | 115-125 cm |
| L | 100-115 cm |
| M | 90-105 cm |

DISPOSAL OF THE PRODUCT
Do not dispose the product in the nature.
Dispose of the product in accordance with
the waste laws of your municipality and/or
country. The product does not present a
risk of contamination.

This packaging can be
recycled as paper and
the bag as plastic.

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EN Stable and compliant back orthosis with extra space for the stomach. To be used during lumbago, sciatica and other usually occurred back problems. The shape of the front section is adjustable on both sides.

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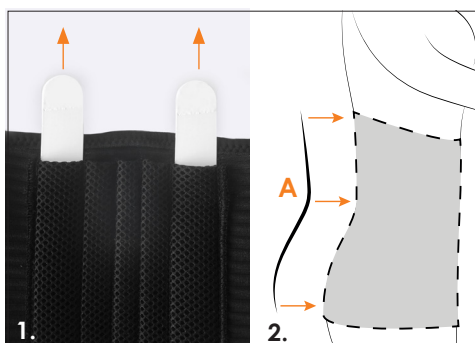
- EN** 1. Pull out the two white stays from the backpockets and shape them in line with the contour of the back.
2. Make sure the stays follow the shape of the back **A** and put them back into the pockets.
3. Loosen the elastic straps **B** and attach to the backpart **C**.
4. Open **C** and put the orthosis in place. Adjust the Velcro closure to the frontpart until the desired shape of the stomach is reached and feels comfortable.
5. Pull the elastic straps **B** firmly outward-forward. Close at front **D**.

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EN Lumbal/lumbosacral back pain.

Contraindications: Unstable back fracture. Sciatica pain of unclear genesis.

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EN To determine the size, measure hip circumference in cm.

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Material Composition

80% Polyester, 9% Spandex, 8% Nylon, 3% Natural rubber latex*.

*Latex is covered does not come in contact with skin.

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EN Machine wash at 40°C (104°F). Remove the stays. Seal the Velcros. Use washing bag. Do not bleach. Do not dry clean. Do not iron. Do not tumble dry.

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EN If a stinging sensation, rash or other changes in skin color occur, discontinue use and contact your doctor or other medically trained person. Only medically trained persons are to prescribe and recommend the use of the product. For best results, follow instructions for use. Periodically check the condition of the Velcro to ensure they are free from grease and damage. Use in combination with other medical products should be supervised by qualified medical practitioner. Serious incidents shall be reported to NordiCare and the competent authority.

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For more information please consult our product catalogue or visit www.nordicare.se. For further questions, contact your doctor, health practitioner or medical equivalent - or NordiCare: +46 42-35 22 20, info@nordicare.se.