Tuff

art no 50200

BACK SUPPORT

LEVEL OF SUPPORT - STRONG

Size	Hip Circumference
2XL	125-135 cm
XL	115-125 cm
L	105-115 cm
M	95-105 cm
S	80-95 cm
XS	65-80 cm

DISPOSAL OF THE PRODUCT
Do not dispose the product in the nature.
Dispose of the product in accordance with
the waste laws of your municipality and/or
country. The product does not present a
risk of contamination.

This packaging can be
recycled as paper and
the bag as plastic.

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EN Extra stable low back support for different body types. Elastic straps for adjustable heavy support. Suitable for pain conditions in the lumbar spine, hip and pelvis. Also to ease residual pelvic pain after pregnancy.

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EN 1. Loosen the elastic straps **A**.

2. Put the back support in place and fasten the Velcro at mid front over the abdomen/hip **B**, without tightening too hard.

3. Pull the elastic straps firmly outward-forward to the desired stability. Close at front **C**.

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EN Lumbal/lumbosacral back pain.

Contraindications: Unstable back fracture. Sciatica pain of unclear genesis.

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EN Machine wash at 40°C (104°F). Seal the Velcros. Use washing bag. Do not bleach. Do not dry clean. Do not iron. Do not tumble dry.

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EN If a stinging sensation, rash or other changes in skin color occur, discontinue use and contact your doctor or other medically trained person. Only medically trained persons are to prescribe and recommend the use of the product. For best results, follow instructions for use. Periodically check the condition of the Velcro to ensure they are free from grease and damage. Use in combination with other medical products should be supervised by qualified medical practitioner. Serious incidents shall be reported to NordiCare and the competent authority.

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EN To determine the size, measure hip circumference in cm. If the orthosis is to be used at waist height, use the waist measurement rather than the hip measurement.

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Material Composition

Polyester 50%, Nylon 30%, Spandex 20%. Latex free.

For more information please consult our product catalogue or visit www.nordicare.se. For further questions, contact your doctor, health practitioner or medical equivalent - or NordiCare: +46 42-35 22 20, info@nordicare.se.