50203 - 32 cm, white

50221 - 16 cm, white

50222 - 20 cm, black

50202 - 25 cm, white

Figur


art no 50202, 50203, 50221, 50222

FIXATION BELT

INSTRUCTIONS FOR USE

Figur – FIXATION BELT

art no 50202, 50203, 50221, 50222

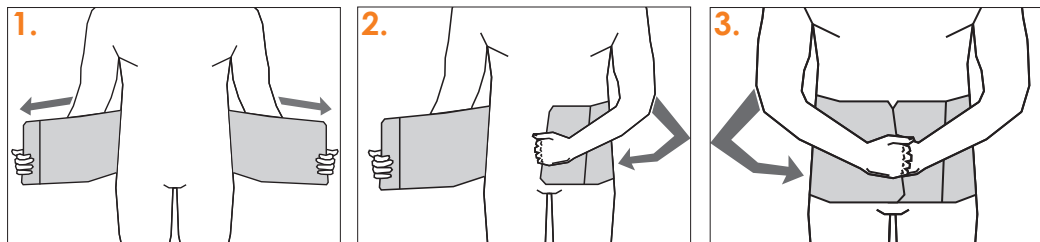Application

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2. Spänn till önskat stöd.

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Size	Circumference intended placement (cm)	Sizes available in heights/variants (cm)
XS	65-85	20
S	80-95	16, 20, 25, 32
M	90-105	16, 20, 25, 32
L	100-115	16, 20, 25, 32
XL	110-125	16, 20, 25, 32
2XL	120-135	16, 20, 25, 32
3XL	130-145	16, 25

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EN Binder for stoma/non-complicated parastomal hernia. To prevent hernia. Binder/fixation band to be used after different types of surgery on abdomen/torso for mobilization after surgery.

Contraindication: Incarcerated hernia.

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EN Machine wash at 60°C (106°F) (the product will last longer if washed at a lower temperature). Seal the Velcros. Use washing bag. Do not bleach. Do not dry clean. Do not iron. Do not tumble dry.

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EN If a stinging sensation, rash or other changes in skin color occur, discontinue use and contact your doctor or other medically trained person. Only medically trained persons are to prescribe and recommend the use of the product. For best results, follow instructions for use. Periodically check the condition of the Velcro to ensure they are free from grease and damage. Use in combination with other medical products should be supervised by qualified medical practitioner. Serious incidents shall be reported to NordiCare and the competent authority.

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For more information please consult our product catalogue or visit www.nordicare.se. For further questions, contact your doctor, health practitioner or medical equivalent - or NordiCare: +46 42-35 22 20, info@nordicare.se.

NordiCare®

NordiCare Ortopedi & Rehab
Solrosvägen 1, SE-263 62 Viken, Sweden.
Tel +46 42 35 22 20, Fax +46 42 35 22 21
info@nordicare.se, www.nordicare.se

STANDARD
100
1976-298
DTI

MD Class I

This packaging can be recycled as paper and the bag as plastic.

DISPOSAL OF THE PRODUCT
Do not dispose the product in the nature. Dispose of the product in accordance with the waste laws of your municipality and/or country. The product does not present a risk of contamination.