Lyft

art no 50206

ABDOMINAL BINDER

2025-12-12

Available color:

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MD Class I

OEKO TEX[®] 100 STANDARD
 1976-298 DTI

This packaging can be recycled as paper and the bag as plastic.

DISPOSAL OF THE PRODUCT
 Do not dispose the product in the nature. Dispose of the product in accordance with the waste laws of your municipality and/or country. The product does not present a risk of contamination.

NO Lyft er et abdominalt elastisk bælte med formet magpartii som gillr behagellig og effektiv støtte/kompression for personer med stor mage.

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EN Lyft is an abdominal elastic girdle with a shaped stomach area that provides comfortable and effective support/compression for people with a larger abdomen.

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EN 1. Position the binder and secure the Velcro fastening at the side with the handle. If you have a hernia, it may be preferable to put the binder on while lying down. This will help the hernia to sink in and ensure that the binder provides optimal support.

2. Adjust until comfortable. The binder should give a comfortable support, make sure it is not too tight.

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EN Mobilization after surgery. Stabilizing girdle for hernia/stoma/ostomy hernia. Prevents herniation. Fixation of stoma bandage. **Contraindication:** Incarcerated hernia.

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EN Lyft is 24 cm high at front and 23 cm high at back. For correct size measure the hip circumference in cm.

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EN If a stinging sensation, rash or other changes in skin color occur, discontinue use and contact your doctor or other medically trained person. Only medically trained persons are to prescribe and recommend the use of the product. For best results, follow instructions for use. Periodically check the condition of the Velcro to ensure they are free from grease and damage. Use in combination with other medical products should be supervised by qualified medical practitioner. Serious incidents shall be reported to NordiCare and the competent authority.

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EN Machine wash at 40°C (106°F). Seal the Velcros. Use washing bag. Do not bleach. Do not dry clean. Do not iron. Do not tumble dry.

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Material Composition

Polyamide 64%, Polyester 30%, Elastane 6%. Latex free.