NordiCare®

# LyftPlus Special

art no 50208

ABDOMINAL BINDER

semi-finished

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**EN** Semi-finished product to be used for manufacturing of an extra supportive elastic abdominal binder. The front is constructed to provide an efficient support and "lift" of the abdomen.

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Phone +46 42 35 22 20,  
Fax +46 42 35 22 21

info@nordicare.se,  
www.nordicare.se

This packaging can be recycled as paper and the bag as plastic.

**DISPOSAL OF THE PRODUCT**  
Do not dispose the product in the nature. Dispose of the product in accordance with the waste laws of your municipality and/or country. The product does not present a risk of contamination.

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**EN** Semi-finished product - Customize and sew the product ready.

1. Position the binder and secure the Velcro fastenings at the sides. If you have a large or hanging abdomen/hernia, it may be preferable to put the binder on while lying down. This will help the abdomen/hernia to settle in the right place and ensure that the binder provides optimal support.
2. Adjust until comfortable. Make sure it is not too tight.

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**EN** Mobilization after surgery. Stabilizing girdle for hernia/stoma/ostomy hernia. Prevents herniation. Fixation of stoma bandage.

**Contraindication:** Incarcerated hernia.

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### Heights and Measurement

LyftPlus Special is 39 cm high at front and 25 cm high at back. For correct size measure the hip circumference in cm.

Size	Hip Circumference
M	90 - 105 cm
L	100 - 115 cm
XL	110 - 125 cm
2XL	120 - 135 cm

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**EN** If a stinging sensation, rash or other changes in skin color occur, discontinue use and contact your doctor or other medically trained person. Only medically trained persons are to prescribe and recommend the use of the product. For best results, follow instructions for use. Periodically check the condition of the Velcro to ensure they are free from grease and damage. Use in combination with other medical products should be supervised by qualified medical practitioner. Serious incidents shall be reported to NordiCare and the competent authority.

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### Material Composition

66% Polyamide, 28% Polyester, 6% Elastane. Latexfree.

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**EN** Machine wash at 40°C (106°F). Seal the Velcros. Use washing bag. Do not bleach. Do not dry clean. Do not iron. Do not tumble dry.

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### Accessory

To be ordered separately, if necessary for individual adaption.

Product	Art.no:
Elastic edge binding 25 mm, Black, Approx. 100 m	91121-00-00
Woven ribbon (for covering stays), 25 mm, Black, 25 m	91123-00-00
Velour Active, Black, 1,5 x 1 m	91130-00-00