NordiCare[®] +

StomiMini

art no 50213




STOMA BINDER

INSTRUCTIONS FOR USE

StomiMini – STOMA BINDER

art no 50213

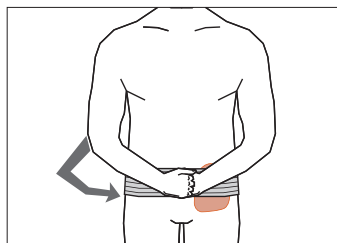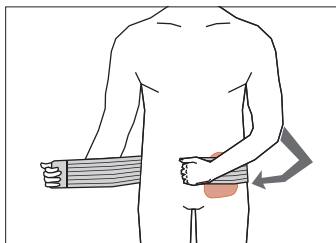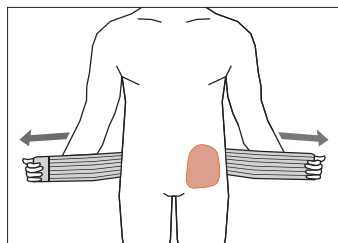Application

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EN 1. Apply the fixation band around the intended placement.
2. Tighten to required support and close the Velcro in front.

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EN Circumference of intended placement. Universal size, fits circumference measuring up to 120 cm.

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EN Machine wash at 60°C (140°F). Seal the Velcros. Use washing bag. Do not bleach. Do not dry clean. Do not iron. Do not tumble dry.

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Material

Polyamide 77%, Elastane 13%, Cotton 10%. Latex Free

Model	Art. no.
StomiMini, White	50213-89-50
StomiMini, Black	50213-80-50

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EN To be used as a stoma band for fixation of ostomy pouches. **Contraindications:** Ostomy hernia.

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EN If a stinging sensation, rash or other changes in skin color occur, discontinue use and contact your doctor or other medically trained person. Only medically trained persons are to prescribe and recommend the use of the product. For best results, follow instructions for use. Periodically check the condition of the Velcro to ensure they are free from grease and damage. Use in combination with other medical products should be supervised by qualified medical practitioner. Serious incidents shall be reported to NordiCare and the competent authority.

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EN Soft fixation band for fixation of ostomy pouch. Discreet and invisible under clothing. One universal size fits circumference up to 120 cm, easy to adjust as the Velcro attaches to the surface of the entire band. Suitable for ostomy on either right or left side. StomiMini is available in black and white. Height 7,5 cm.

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For more information please consult our product catalogue or visit www.nordicare.se. For further questions, contact your doctor, health practitioner or medical equivalent - or NordiCare: +46 42-35 22 20, info@nordicare.se

NordiCare

Available colors:

MD Class I

STANDARD 100
1976-298
DTI

NordiCare Ortopedi & Rehab
Solrosvägen 1, SE-263 62 Viken, Sweden.
Tel +46 42 35 22 20, Fax +46 42 35 22 21
info@nordicare.se, www.nordicare.se

This packaging can be recycled as paper and the bag as plastic.

2024-12-17