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
Balder


art no 50216

HERNIA BELT

2026-03-06

Size	Hip Circumference
XS	70 - 85 cm
S	80 - 95 cm
M	90 - 105 cm
L	100 - 115 cm
XL	110 - 125 cm

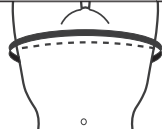DISPOSAL OF THE PRODUCT
Do not dispose the product in the nature. Dispose of the product in accordance with the waste laws of your municipality and/or country. The product does not present a risk of contamination.

This packaging can be recycled as paper and the bag as plastic.

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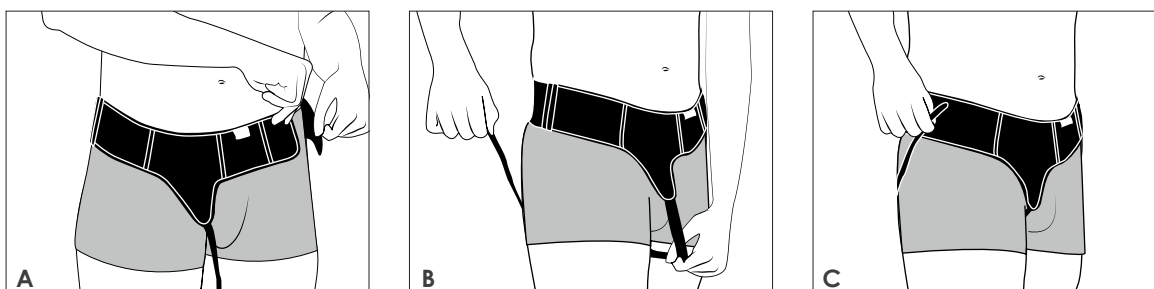
EN Balder is a discreet inguinal hernia support belt, well covered underneath clothing. The belt provides good support and is suitable for smaller to medium hernias. Available in bilateral and unilateral design.

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EN 1. Fit the hip belt **A** into place. Adjust for steadiness and comfort.
2. Take the long strap **B** and pull it back between legs. Pull up over the hip and attach **C**. If the strap is too long, open the Velcro attachment **D** and cut the strap to desired length.
3. Adjust for comfort and sufficient of pressure. Included pad **E** should be placed in the pocket. Using boxer shorts with the belt increases comfort and minimizes the risk of soreness.

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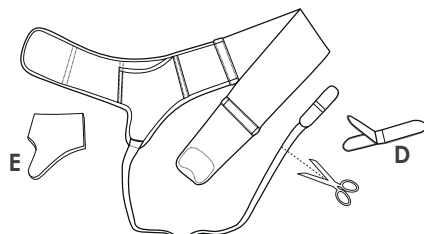
EN If a stinging sensation, rash or other changes in skin color occur, discontinue use and contact your doctor or other medically trained person. Only medically trained persons are to prescribe and recommend the use of the product. For best results, follow instructions for use. Periodically check the condition of the Velcro to ensure they are free from grease and damage. Use in combination with other medical products should be supervised by qualified medical practitioner. Serious incidents shall be reported to NordiCare and the competent authority.

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50216 Balder Unilateral: Part D+E x 1 is included.

Material

80% Polyamide, 14% Polyester, 6% Natural rubber latex*.
Pad: EVA foam.

*Latex is covered does not come in contact with skin.

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EN Inguinal hernia. Pre- or post op or with conservative treatment. **Contraindications:** Incarcerated inguinal hernia.

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EN In order to determine the correct size, measure the hip circumference at its widest point.

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EN Machine wash at 40°C (104°F). Seal the Velcros. Use washing bag. Do not bleach. Do not dry clean. Do not iron. Do not tumble dry.

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For more information please consult our product catalogue or visit www.nordicare.se. For further questions, contact your doctor, health practitioner or medical equivalent - or NordiCare: +46 42-35 22 20, info@nordicare.se.