
Vira x2

art no 50228

BINDER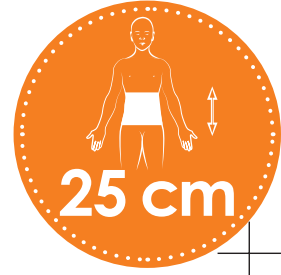


Size	Hip cm	Waist cm
XXS	65-80	50-65
XS	70-90	60-80
S	80-100	70-90
M	90-110	80-100
L	100-120	90-110
XL	110-130	100-120

Available colors: 

DISPOSAL OF THE PRODUCT
Do not dispose the product in the nature. Dispose of the product in accordance with the waste laws of your municipality and/or country. The product does not present a risk of contamination.

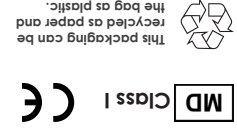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EN Place the binder over the surged area. Begin by fastening the lower band 1 at the bottom of the hip. Then fasten the top band 2. Adjust until it feels comfortable. The girdle should sit tight but not uncomfortably tight.

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EN If a stinging sensation, rash or other changes in skin color occur, discontinue use and contact your doctor or other medically trained person. Only medically trained persons are to prescribe and recommend the use of the product. For best results, follow instructions for use. Periodically check the condition of the Velcro to ensure they are free from grease and damage. Use in combination with other medical products should be supervised by qualified medical practitioners. Serious incidents shall be reported to NordiCare and the competent authority.

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Material

Polyester, 88%, Elastane 12%. Latex free.

For more information please consult our product catalogue or visit www.nordicare.se. For further questions, contact your doctor, health practitioner or medical equivalent - or NordiCare: +46 42-35 22 20, info@nordicare.se.

NordiCare

2025-08-06

NordiCare Ortopedi & Rehab AB
Solrosvägen 1, SE-263 62 Viken, Sweden
Phone +46 42 35 22 20, Fax +46 42 35 22 21
info@nordicare.se, www.nordicare.se

MD Class I CE