

AbdoBas

art no 50236

SURGICAL BINDER

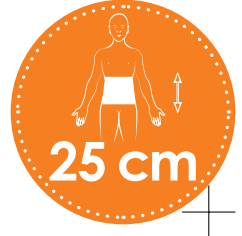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2025-04-15

NL Omtrek van de beoogde locatie.
ES Medidas del perimetro en la ubicación prevista.
PT Medida da circunferência do local previsto.
IT Effettuare la misurazione perimetrale della posizione prevista.
FR Circonférence de l'emplacement envisagé.

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EN If a stinging sensation, rash or other changes in skin color occur, discontinue use and contact your doctor or other medically trained person. Only medically trained persons are to prescribe and recommend the use of the product. For best results, follow instructions for use. Periodically check the condition of the Velcro to ensure they are free from grease and damage. Use in combination with other medical products should be supervised by qualified medical practitioner. Serious incidents shall be reported to NordiCare and the competent authority.

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Material Composition

Cotton 37%, Polyester 20%, Polyester foam 19%, Spandex 15%, Nylon 9%. Latex free.

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EN Can be washed at 95°C (203°F) approx. 10 times. The lifetime of the product increases when washed at 40-60°C (104°F-140°F). Seal the Velcro. Use washing bag. Do not use bleach or softener. Do not dry clean. Do not iron. Do not tumble dry.

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For more information please consult our product catalogue or visit www.nordicare.se. For further questions, contact your doctor, health practitioner or medical equivalent - or NordiCare: +46 42-35 22 20, info@nordicare.se.