## Multi

art no 50240

### BACK SUPPORT

LEVEL OF SUPPORT - MEDIUM

2025-04-07

Size	Hip Circumference
S	80-95 cm
M	90-105 cm
L	105-115 cm
XL	115-125 cm
2XL	125-135 cm

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**EN** 1. Loosen elastic straps.

2. Put the back support in place and fasten the Velcro **A** at mid front over the abdomen/hip, without tightening too hard.

3. Pull the straps **B** firmly outward-forward to the desired stability. Close at front.

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**EN** Lumbal/lumbosacral back pain.

**Contraindications:** Unstable back fracture. Sciatica pain of unclear genesis.

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**EN** To determine the size, measure hip circumference in cm. If the orthosis is to be used at waist height, use the waist measurement rather than the hip measurement.

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### Material Composition

Polyester 42%, Polyamide 40%, Elastic 13%, Polyurethane 5%. Latex free.

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**EN** Machine wash at 40°C (106°F). Seal the Velcros. Use washing bag. Do not bleach. Do not dry clean. Do not iron. Do not tumble dry.

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**EN** If a stinging sensation, rash or other changes in skin color occur, discontinue use and contact your doctor or other medically trained person. Only medically trained persons are to prescribe and recommend the use of the product. For best results, follow instructions for use. Periodically check the condition of the Velcro to ensure they are free from grease and damage. Use in combination with other medical products should be supervised by qualified medical practitioner. Serious incidents shall be reported to NordiCare and the competent authority.

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