Bure Hög

art no 50242

BACK SUPPORT

LEVEL OF SUPPORT - STRONG

2026-03-06

MD Class I

Size	Hip Circumference
S	80-95 cm
M	90-105 cm
L	100-115 cm
XL	115-125 cm
2XL	125-135 cm

DISPOSAL OF THE PRODUCT
Do not dispose the product in the nature.
Dispose of the product in accordance with
the waste laws of your municipality and/or
country. The product does not present a
risk of contamination.

This packaging can be
recycled as paper and
the bag as plastic.

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- EN** 1. Pull out the two white stays from the backpockets and shape them in line with the contour of the back.
2. Make sure the stays follow the shape of the back **A** and put them back into the pockets.
3. Loosen the elastic straps **B**. Put the back support in place and fasten the Velcro at mid front over the abdomen **C**, without tightening too hard.
4. Pull the elastic straps firmly outward-forward to the desired stability. Close at front **D**.

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EN Lumbal/lumbothoracal back pain.
Contraindications: Unstable back fracture. Sciatica pain of unclear genesis.

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EN To determine the size, measure hip circumference in cm. Bure is available in two different heights. The low model fits most, choose the high model if you are tall or if your inconveniences/pain are placed further up on the back.

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Material Composition

Polyester 55%, Polyamide 30%, Spandex 15%. Latex free.

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EN Machine wash at 40°C (104°F). Remove the splints. Seal the Velcros. Use washing bag. Do not bleach. Do not dry clean. Do not iron. Do not tumble dry.

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EN If a stinging sensation, rash or other changes in skin color occur, discontinue use and contact your doctor or other medically trained person. Only medically trained persons are to prescribe and recommend the use of the product. For best results, follow instructions for use. Periodically check the condition of the Velcro to ensure they are free from grease and damage. Use in combination with other medical products should be supervised by qualified medical practitioner. Serious incidents shall be reported to NordiCare and the competent authority.

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For more information please consult our product catalogue or visit www.nordicare.se. For further questions, contact your doctor, health practitioner or medical equivalent - or NordiCare: +46 42-35 22 20, info@nordicare.se