
Smidig

art no 50247

BINDER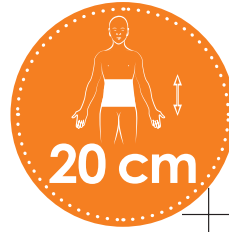

MD Class I

info@nordicare.se
www.nordicare.se

Phone +46 42 35 22 20,
Fax +46 42 35 22 21

Solovägen 1,
SE-263 62 Viken
Sweden

NordiCare

Size	Circumference
XS	60-75 cm
S	70-85 cm
M	80-100 cm
L	95-115 cm
XL	110-125 cm
2XL	120-135 cm

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EN If a stinging sensation, rash or other changes in skin color occur, discontinue use and contact your doctor or other medically trained person. Only medically trained persons are to prescribe and recommend the use of the product. For best results, follow instructions for use. Periodically check the condition of the Velcro to ensure they are free from grease and damage. Use in combination with other medical products should be supervised by qualified medical practitioner. Serious incidents shall be reported to NordiCare and the competent authority.

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Material Composition

28% Polyester, 20% Spandex, 48% Cotton, 4% Nylon. Latex free.

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EN Machine wash at 40°C (104°F). Can be washed at a higher temperature if necessary, max. 60°C (140°F). The product will last longer if washed at a lower temperature. Use a laundry bag. Dry flat. Do not tumble dry. Seal the Velcros before washing. Do not use bleach, chlorine bleach or fabric softener as this could impair the function of the material.

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For more information please consult our product catalogue or visit www.nordicare.se. For further questions, contact your doctor, health practitioner or medical equivalent - or NordiCare: +46 42-35 22 20, info@nordicare.se.