Freja

art no 50254

COMPRESSION / HERNIA BRIEFS

2025-08-05

DISPOSAL OF THE PRODUCT
Do not dispose the product in the nature. Dispose of the product in accordance with the waste laws of your municipality and/or country. The product does not present a risk of contamination.

This packaging can be recycled as paper and the bag as plastic.

Italy
Via Adamello 13
10093 Collegno (TO)

Made by:
SAFTE Srl.

NordiCare Ortopedi & Rehab
Soforsvägen 1
SE-263 62 Viken, Sweden
Phone +46 42 35 22 20,
Fax +46 42 35 22 21
info@nordicare.se,
www.nordicare.se

NordiCare®

EN Comfortable hernia briefs in women's fit with hook and eye opening. Designed in soft cotton material, provides firm compression. Used for conservative treatment of mild or medium sized inguinal hernia. Also postoperatively for faster mobilization, pain relief and to prevent relapse. Freja is designed with pockets on both sides for pads. Flat pads are included.

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EN To be worn instead of regular briefs. Flat pads (1) are included, convex pads (2) for increased pressure can be ordered separately. The trousers provides lift and support (3).

- Place the pads in the pockets. Convex pads should be positioned with the convex part against the body. Remove pads before washing briefs.
- Put on the briefs first thing in the morning. Preferably laying down, when the hernia is most sunken.
- The fly is easily opened with velcro closure, when going to the toilet.
- To be worn directly against the skin. Do not use the briefs directly on damaged/irritated skin.
- Do not use with lotions or creams.

Materials

Cotton 86%, Elastane 14%. Pads in Polyethylene.
Pads in Polyethylene. Latex free.

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EN Machine wash at 40°C (104°F). Seal the Velcros. Use washing bag. Do not bleach. Do not dry clean. Do not tumble dry.

For more information please consult our product catalogue or visit www.nordicare.se. For further questions, contact your doctor, health practitioner or medical equivalent - or NordiCare: +46 42-35 22 20, info@nordicare.se

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EN Only medically trained persons are to prescribe and recommend the use of the product. For best results, follow instructions for use. Use in combination with other medical products should be supervised by qualified medical practitioners. Serious incidents shall be reported to NordiCare and the competent authority.

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EN For conservative treatment of light/medium inguinal hernia and postoperatively to support the area of surgery and provide pain relief. Also after abdominal surgery/liposuction without pads.
Contraindication: Incarcerated hernia.

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EN Waste circumference (B) in cm. Height is measured from the pubis and up (A).

Size	A	B
02	20	65-69
03	20	70-74
04	20	75-79
05	20	80-84
06	20	85-89
07	20	90-94
08	20	95-99