**INTRODUKTION**

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**INTRODUCTION**

The Pectus Carinatum Orthosis was ordered by your physician for you as an aid in the reduction or correction of an abnormal deformation of the chest.

It provides increased anterior/posterior-directed and compression corrective forces for the reduction of Pectus Carinatum and other anomalies.

**Indications**

- Abnormal shape of the chest
- A protrusion of both the sternum (breast bone) and the cartilage portion of the ribs
- Bowed Chest (Pigeon breast)

**Contraindicated**

- Severe deformations

**Application**

- Follow all instructions carefully that were given to you by the fitting specialist and protocols set by your prescribing physician.
- When removing and applying the orthosis only use one side of the ratchet buckle to tighten and remove.
- The ratchet can be loosened by pushing down on the ratchet release lever under the ratchet for removal.
- Place the ladder strap into the ratchet and tighten.

**Care of the Orthosis**


- It is recommended that a snug fitting under shirt be worn between the skin and the Orthosis.
- The foam pads can be washed by hand with mild soap and water and towel dried.

 Do not use heat to dry or place near any heating device.


**Any questions or problems**

- For any physical problems, call your prescribing physician.
- Call your fitting specialist for any problems associated with the orthosis.

Front View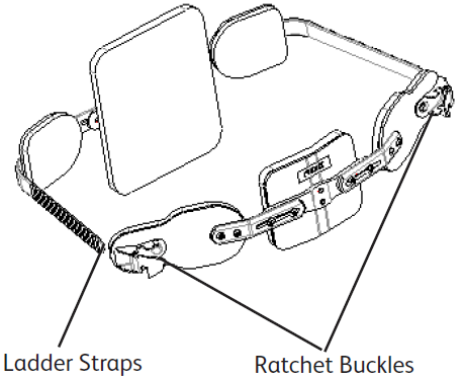

Back View

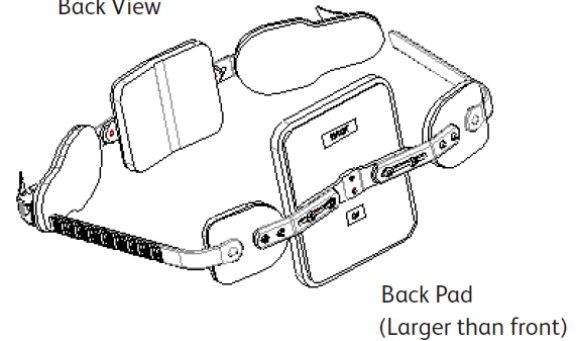**Special Instructions:**

---

---

---

---

---

---

---

## FITTING SPECIALIST INSTRUCTIONS INSTALLATION AND USE



**NOTE:** This device should be fit by a qualified fitting specialist to the protocols of the prescribing physician.

### Step 1: Measurements and Initial-Adjustments

- Take a circumferential measurement at or about the xiphoid.
- Loosen the sternal and posterior adjustment screws to increase or decrease circumference by sliding components and tighten.
- The anterior and posterior pads can be removed for customizing the shape for each individual's needs.

### Step 2: Contouring metal components

- The metal components can be contoured with the use of bending irons.
- If bending irons are necessary, we recommend only round edged bending irons to be used. Care should be taken not to make sharp bends in slotted areas.
- Apply orthosis and tighten to desired tension by means of the ratchet buckles.
- For removal, have patient push down on the ratchet release to loosen the ratchet and then slide the strap out of the buckle.
- The patient should be instructed to only remove the orthosis by loosening just one side which ever they prefer.
- Instruct patient how to insert ladder strap into ratchet buckle and tighten to desired tension.

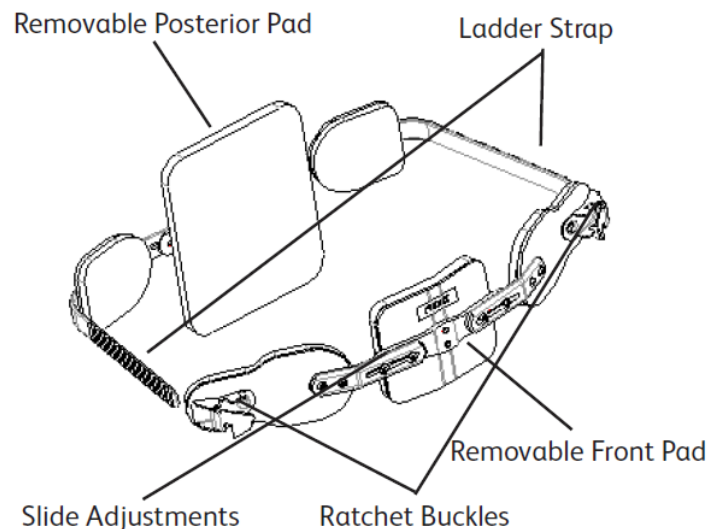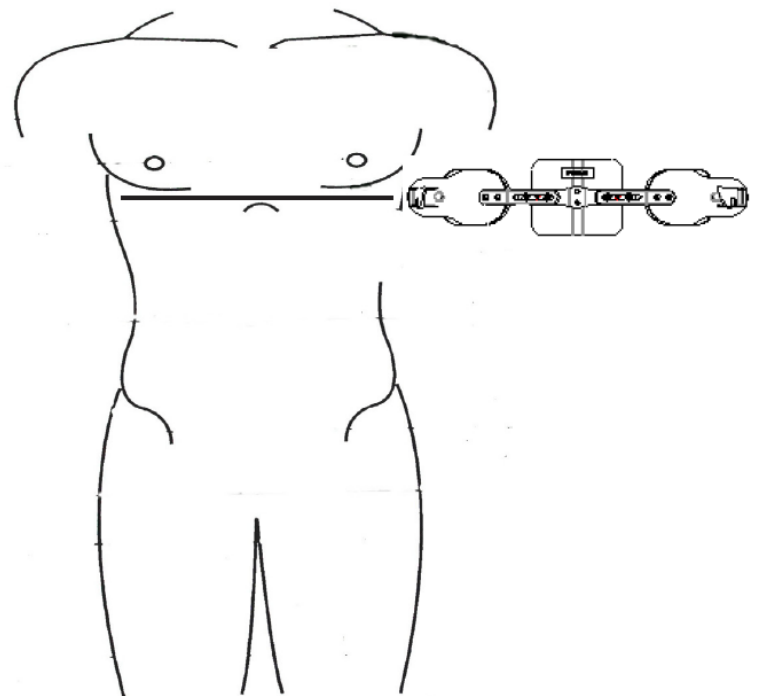### STORAGE AND USE

There are no identified restrictions on the temperature of storage and use.

### DISPOSAL

There are no hazardous materials in the device. Comply with local and national laws and regulations.

Trulife [www.trulife.com](http://www.trulife.com)

Page 2 of 3

## INFORMATION

**MD** This is a Class I medical device. The product is subject to the applicable regulatory requirements for medical devices in the EU. Serious incidents are reported to NordiCare and the competent authority. NordiCare is the importer for the product and forwards any serious incidents to the specified authorized EU representative/legal manufacturer. Prescription, fitting and recommendation of use of this product should be done by, or under the guidance of a medically trained person.

 This product is restricted to a "Single Patient Use Only". Do not reuse for multiple patients.

### Trulife

3225 Woburn St. Ste160  
 Bellingham, WA. 98226  
 USA

**Phone** (+1) 360 697 5656

**Email** [supportop@trulife.com](mailto:supportop@trulife.com)

### MDSS GmbH

Schiffgraben 41  
 30175 Hannover  
 Germany

**Phone** (+49) 511 6262 8630

**Importer:** **NordiCare<sup>+</sup>**  
**NordiCare Ortopedi & Rehab AB**  
 Solrosvägen 1  
 263 62 Viken  
 Sweden  
**Phone** (+46) 42 35 22 20  
[info@nordicare.se](mailto:info@nordicare.se), [www.nordicare.se](http://www.nordicare.se)