50531 - Låg

50532 - Mellan

50533 - Hög

# Ronja

art no 50531, 50532, 50533, 50536

## SEMI REGID BACK SUPPORT

LEVEL OF SUPPORT - STRONG

DISPOSAL OF THE PRODUCT  
Do not dispose the product in the nature.  
Dispose of the product in accordance with  
the waste laws of your municipality and/or  
country. The product does not present a  
risk of contamination.

This packaging can be  
recycled as paper and  
the bag as plastic.

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**EN** Stable back support in thin firm material. Elastic straps and moldable stays for adjustable support. Suitable for more severe back pain, disc herniation and after surgery. Available in three heights.

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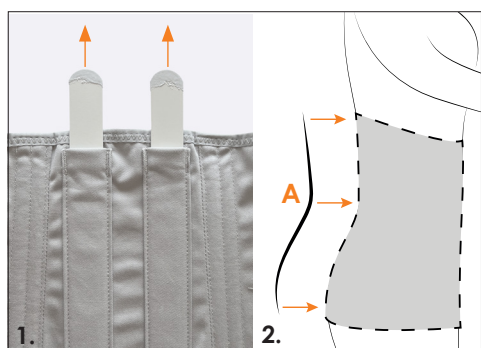
**EN** 1. Pull out the two white stays from the backpockets and shape them in line with the contour of the back.  
2. Make sure the stays follow the shape of the back (A) and put them back into the pockets.  
3. Loosen all elastic straps (B). Put the orthosis in place and fasten the Velcro at mid front over the abdomen (C), without tightening too hard.  
4. Pull the lower elastic straps (B1) firmly outward-forward. Repeat with the upper straps (B2).

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**EN** To determine the size, measure hip circumference in cm. Straight body shape: go up one size.

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**EN** Ronja Låg/Mellan: Lumbar/lumbosacral back pain.

**Ronja Hög:** Lumbar/lumbothoracal back pain.

**Contraindications:** Unstable fracture of the back. Sciatica pain of unclear genesis.

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**EN** Machine wash at 40°C (104°F). Remove the stays. Seal the Velcros. Use washing bag. Do not bleach. Do not dry clean. Do not iron. Do not tumble dry.

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### Material composition

69% Cotton, 14% Polyester, 9% Spandex, 8% Polyamide. Latex free.

### Heights

Ronja is available in three different heights and in a semi-finished model (Ronja Kit) with edging, elastic stripe and splints included, to finish the product.

Model	Art.no	Height front	Height back
Ronja Låg	50531	18 cm	28 cm
Ronja Mellan	50532	20 cm	33 cm
Ronja Hög	50533	23 cm	38 cm
Ronja Kit	50536	22 cm	37 cm

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**EN** If a stinging sensation, rash or other changes in skin color occur, discontinue use and contact your doctor or other medical-trained person. Only medically trained persons are to prescribe and recommend the use of the product. For best results, follow instructions for use. Periodically check the condition of the Velcro to ensure they are free from grease and damage. Use in combination with other medical products should be supervised by qualified medical practitioners. Serious incidents shall be reported to NordiCare and the competent authority.

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