
NordiCare®

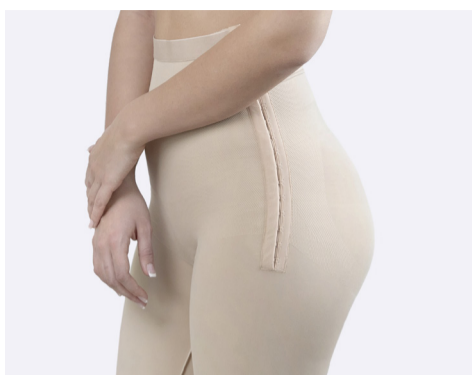
Compression Garments by NordiCare

Patsy

art no 51201

FULL LENGTH PANTS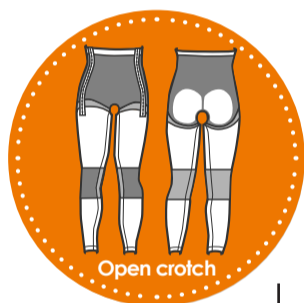

Compression class 1 (15-21 mmHg)


DISPOSAL OF THE PRODUCT
Do not dispose the product in the nature.
Dispose of the product in accordance with
the waste laws of your municipality and/or
country. The product does not present a
risk of contamination.


This packaging can be
recycled as paper and
the bag as plastic.

Available colors:
[Black swatch] [Grey swatch]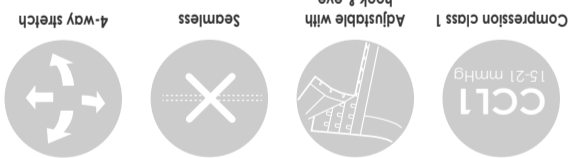

Size	Waist cm	Hip cm	Thigh cm
XS	62-68	86-91	46-50
S	69-75	92-97	50-54
M	76-83	98-103	54-58
L	84-92	104-111	58-62
XL	93-101	112-119	62-66
2XL	102-110	120-127	66-70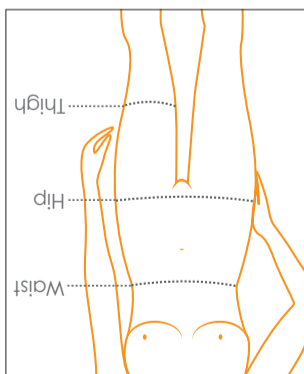

NordiCare®
Sofsvägen 1,
SE-263 62 Viken
Sweden
Phone +46 42 35 22 20,
Fax +46 42 35 22 21
info@nordicare.se,
www.nordicare.se

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EN Full length compression pants. After surgery on the leg or abdomen. Closure on both sides, open crotch and silicone waistband.

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EN 1. Make sure your skin is dry. Open the hook and eye closure.

2. Carefully pull the pants so that the "diamond knitting" is positioned over/around the knee.

3. Pull the pants up to the hip. Make sure that the crotch part is high/ comfortable in the crotch and that the pants are centered at the back. Then pull your pants over your stomach and back. To facilitate fold the silicone edge outwards.

4. Close the hook/eyes in the side at the desired size/shape/compression. Make sure the flap behind the hook/eyes is flat against the skin.

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EN If you experience a stinging sensation, chafing or color changes in your skin, remove the product and contact your medical professional. Fitting, and recommended use should be handled by or under the supervision of a medical professional. For best results, follow the user instructions. Keep in mind that measurements may vary during the healing process, a time after surgery when any swelling has gone down, a smaller size may be required. Serious incidents shall be reported to NordiCare and the competent authority.

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87% Polyamide, 13% Elastane. Latex Free.

Washing Instructions

For more information please consult our product catalogue or visit www.nordicare.se. For further questions, contact your doctor, health practitioner or medical equivalent - or NordiCare: +46 42-35 22 20, info@nordicare.se.