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EN 1. Make sure your skin is dry. Open the hook and eye closure.

2. Carefully pull the body up to the hip. Make sure that the crotch part is high/comfortable in the crotch and that the body is centered at the back.

3. Then pull the body over your abdomen and back, make sure to pull your back properly so that it gets high before you tighten the shoulder straps.

4. Close the hook/eyes in the side at the desired size/shape/compression. Make sure the flap behind the hook/eyes is flat against the skin.

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EN If you experience a stinging sensation, chafing or color changes in your skin, remove the product and contact your medical professional. Fitting, and recommended use should be handled by or under the supervision of a medical professional. For best results, follow the user instructions. Keep in mind that measurements may vary during the healing process, a time after surgery when any swelling has gone down, a smaller size may be required. Serious incidents shall be reported to NordiCare and the competent authority.

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Washing Instructions

140 F

For more information please consult our product catalogue or visit www.nordicare.se. For further questions, contact your doctor, health practitioner or medical equivalent - or NordiCare: +46 42-35 22 20, info@nordicare.se.