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**EN** 1. Before use, ensure your skin is dry. Open the hook-and-eye closure.

2. Gently pull on one sleeve at a time. Make sure the bolero is centered across the back, and adjust it so that the structured knitting in the middle of each sleeve is aligned with the elbow crease.

3. Fasten the hook-and-eye closure at the desired size and compression. Ensure the flap behind the closure lies flat against the skin. The fit of the bolero should be tight but should not feel uncomfortable.

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**EN** Should you experience any stinging, irritation, or discoloration of the skin during use of the product, remove the product immediately and contact the supplier. The fitting and recommendation for use of the product must always be conducted by a physician or under the supervision of authorized healthcare professionals. Please note that measurements may vary throughout the healing process following surgery. As swelling subsides, it may become necessary to select a smaller size. Adhere to the instructions for use to ensure optimal results. All serious adverse events related to the use of the product must be reported to NordiCare and the appropriate regulatory authorities.

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87% Polyamide, 13% Elastane. Latex Free.

## Washing Instructions

For more information please consult our product catalogue or visit [www.nordicare.se](http://www.nordicare.se). For further questions, contact your doctor, health practitioner or medical equivalent - or NordiCare: +46 42-35 22 20, [info@nordicare.se](mailto:info@nordicare.se)