
Compression Garments by NordiCare

Jolene

art no 51213

BODY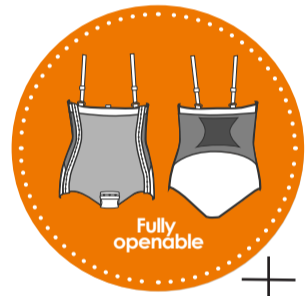

Compression class 1 (15-21 mmHg)

DISPOSAL OF THE PRODUCT
Do not dispose the product in the nature. Dispose of the product in accordance with the laws of your municipality and/or country. The product does not present a risk of contamination.

This packaging can be recycled as paper and the bag as plastic.

- Available colors:
- Compression class 1 (15-21 mmHg)
- Seamless
- 4-way stretch
- Fully adjustable & adjustable with hook & eye

Size	Waist cm	Hip cm
XS	62-68	86-91
S	69-75	92-97
M	76-83	98-103
L	84-92	104-111
XL	93-101	112-119
2XL	102-110	120-127

NordiCare
Sölvägen 1, SE-263 62 Viken, Sweden
Phone +46 42 35 22 20, info@nordicare.se, www.nordicare.se

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EN Compression body with high waist. After surgery on the abdomen. Closes at the sides and at the crotch. Adjustable shoulder straps.

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EN 1. Make sure the skin is dry. Open the hook and eye closure.

2. Carefully pull the body over the abdomen and back. Make sure the body is centered at the back.

3. Make sure to pull your back properly so that it gets high before you tighten the shoulder straps.

4. Close the hook/eyes in the side to achieve desired size/shape/compression. Make sure the flap behind the hook/eyes is flat against the skin.

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EN Should you experience any stinging, irritation, or discoloration of the skin during use of the product, remove the product immediately and contact the supplier. The fitting and recommendation for use of the product must always be conducted by a physician or under the supervision of authorized healthcare professionals. Please note that measurements may vary throughout the healing process following surgery. As swelling subsides, it may become necessary to select a smaller size. Adhere to the instructions for use to ensure optimal results. All serious adverse events related to the use of the product must be reported to NordiCare and the appropriate regulatory authorities.

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Washing Instructions

For more information please consult our product catalogue or visit www.nordicare.se. For further questions, contact your doctor, health practitioner or medical equivalent - or NordiCare: +46 42-35 22 20, info@nordicare.se.