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EN Make sure the Velcro is free from grease and that the adhesive-ability is not deteriorated. Regularly control both condition and function of the straps and plastic buckles.

Skin care:

- Make sure that there is no pressure or skin irritation by the pressure points.
- It is important to check the baby's skin at least once a day.
- Take particularly in account the following areas; shoulders, neck, back, chest, hip, back of knees, legs and ankles.
- Use of ointments or powder should be made in consultation with medical adviser.
- For better comfort and hygiene, as well as to protect the skin, it is advisable to use trousers with long legs of cotton under the orthosis.

Advice for bathing:

- The orthosis should not be used when the baby is bathing. Therefore bathe the baby as short time as possible to minimize the time without the orthosis in place.
- Gently Massage Baby's skin at the pressure points to relax the muscles.

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For more information please consult our product catalogue or visit www.nordicare.se. For further questions, contact your doctor, health practitioner or medical equivalent - or NordiCare: +46 42-35 22 20, info@nordicare.se

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EN Hipabduction orthosis used to position the hip joints of babies with hip dysplasia.

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MD Class I **CE**

This packaging can be recycled as paper and the bag as plastic.

DISPOSAL OF THE PRODUCT
Do not dispose the product in the nature. Dispose of the product in accordance with the waste laws of your municipality and/or country. The product does not present a risk of contamination.

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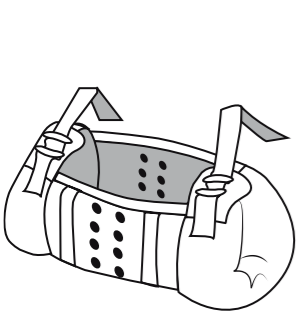
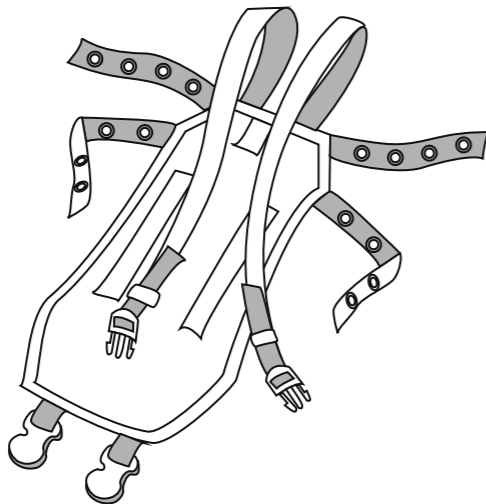
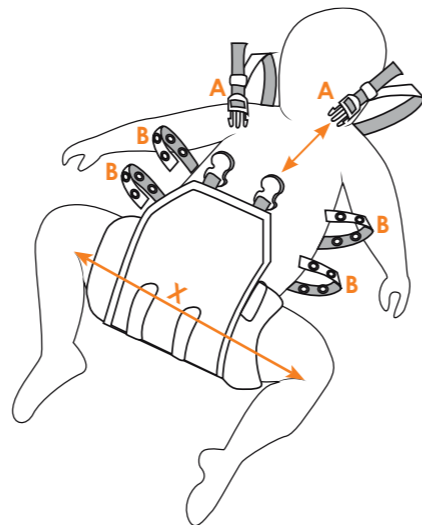
3. Application

Measurements

Size	Width, plastic orthosis, cm	Measurements between the knees, cm (X)
X-Small	14	15-18
Small	16	17-20
Medium	18	19-22
Large	20	21-24
X-Large	22	23-26
2X-Large	24	25-28

Instruction for fitting/Application

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EN 1. Put on the plastic orthosis (1) as a diaper with the child's hip joint in the desired abduction.

2. Fix the plastic orthosis with the textile pant (2) by attaching the pants against corresponding Velcro bands on the plastic orthosis. The long shoulder straps should be positioned against the child's back.

3. Fasten the shoulder straps A in the plastic buckle at the belly. Adjust the straps to the desired length.

4. Adjust the four side straps B over the belly and fasten with the push buttons.

För the brace to function properly it is important that the correct size is used. Do not push the brace too tight against the child's thighs, the skin is not to be scratched or irritated at the pressure points.

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EN Congenital hip dislocation/subluxation. Hip dysplasia. **Contraindications:** Dislocated hip.

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EN Physician or medical professional is to determine the size and how long the brace should be used. The size of the orthosis should be exchanged as the child grows. The ordination of how long the brace should be used per day must be followed. A brace fitted on a child must be checked by the responsible caregiver on a regular basis. The brace should never be used for other use than the intended. Do not let children play with it. The brace should not be used when the child is being bathed or showered. Ointments or powders should only be used after consultation with physician. Check the velcro straps being free from dirt and that the adhesive has not deteriorated. Regularly check the tape and plastic buckles being intact and functioning.

Prescription, initial fitting and recommendation on how to use this product should be made by medically trained person. If changes in color of skin or other blood circulation related problems and skin irritation occurs, immediately remove the brace. The brace is only for one patient use. Serious incidents shall be reported to NordiCare and the competent authority.

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Material Composition

Polyethylene 55%, Polyester 24%, Cotton 17 %, Polyamide 4%. Latex free.

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EN The Pant is machine washable at 40 degrees. Wipe the plastic orthosis with a soft damp cloth, use mild detergent.

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