User information and care

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EN The prescription, recommendation of use and fitting of this product should only be made by a healthcare professional or orthotic fitter. Serious incidents shall be reported to NordiCare and the competent authority.

Use the knee brace according to your fitter's instructions: Always read carefully all the instructions before using this product. It is important to follow the doctor's/fitter's instructions during the entire rehabilitation period, especially with regards to the flexion/extension stop settings. Never change the settings without the advice of your doctor/fitter.

Maintenance: Regularly check that the Velcro straps are free from dirt and that their fastening capabilities have not deteriorated. Avoid using skin cream to prevent a negative impact on the bandage. Do not use the knee brace in the bath or shower.

Special attention should be exercised if the knee brace is used by children: In such a case, both the product and its adjustments should be checked regularly by an adult. The same applies if the product is used by a person who cannot monitor the product and adjustments himself/herself due to a physical or mental disability.

Always contact your doctor/fitter: In case of doubt, if you have questions, if you are taking medication, or if you experience side effects such as skin irritation.

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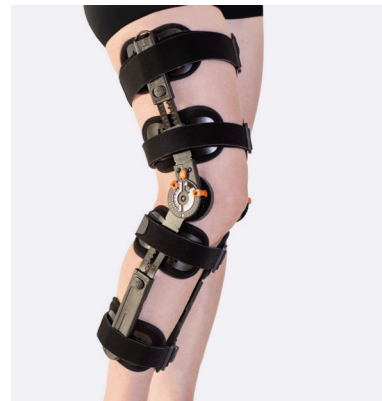
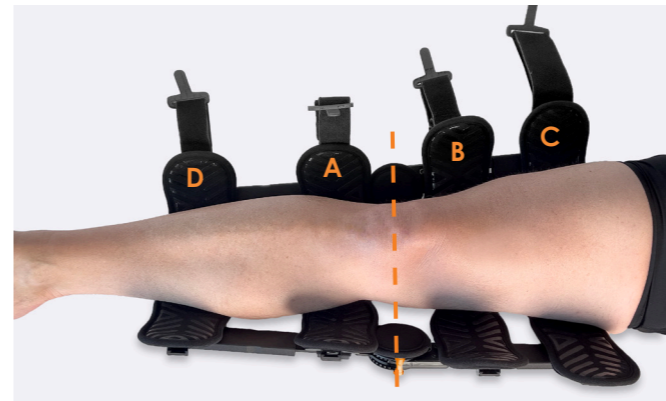
art no 70646

POST-OP KNEE BRACE

For more information please consult our product catalogue or visit www.nordicare.se. For further questions, contact your doctor, health practitioner or medical equivalent - or NordiCare: +46 42-35 22 20, info@nordicare.se.

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- EN 1.** Open up the knee brace and position it around the leg.
- 2.** Set the desired length of the stays with help of the telescopic mechanism. Ensure that the stays are of equal length on both sides. If necessary, the stays can be bent further in order to follow the contours of the leg. **N.B.** Bend in the aluminium stays.
- 3.** Place the knee brace over the centre of the knee joint. Then secure and tighten the strap just below the knee. Repeat with the strap immediately above the knee, followed by the uppermost strap and lastly the lower strap. The strap may be cut to desired length. Make sure the patient is not experiencing any discomfort.

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- EN** With the permission of your healthcare professional, you can remove the knee brace and reposition it by following these instructions:

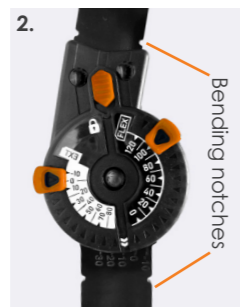
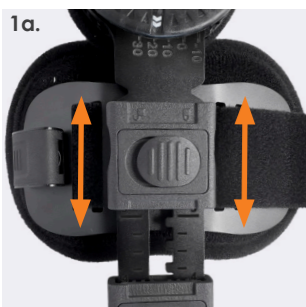
1. Place the knee brace over the centre of the knee.
2. Fasten the strap just below the knee (A), then the strap immediately above the knee (B). Continue by fastening the uppermost strap (C), then finally the lower strap (D).

N.B. Never change the settings without the permission of your orthotic fitter.

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- EN 1.** Adjust the leg cuffs at desired position (1a). The aluminium cuffs can be shaped (1b).
- 2.** The stays can be bent if necessary. Place the joint on a stable surface and bend stays gently but firmly.
- 3.** The joint can be locked with the quick-release button from -10° to 30° at the time of fitting. When the button is unlocked, the patient can move the joint within the set range of motion. Repeat the adjustments on both sides.
- 4.** Set the desired range of motion in the joint: pull the adjustment button and slide to the desired position, release the button. Repeat the setting on both sides.

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- EN** **Textile padding:** Wash 40°C, for best durability hand wash is recommended, dry flat. **The stays and joints:** Wash with a cloth and warm water/mild disinfectant.

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- EN** **Indications:** Postoperative and posttraumatic immobilization/stabilization of the knee joint. **Contraindications:** Not intended for people with weight >110 kg.

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Material Composition:

Aluminium, Polyoxymethylene, Polyamide, Polyurethane, Viscose, Elastane, Silicone, Latex free.

Size:

SE Universalstorlek.

DK Universalstørrelse.

NO Universell størrelse.

FI Yksi koko.

EN Universal.

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