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### English Universal Ankle Stirrup

### Indications

- Severe and moderate acute lateral and medial ankle sprains (functional and post-operative treatment)

- Malleolar fracture Weber A (in the rehabilitative phase)

- Post-Op for plated fractures

### Precautions

- Do not apply the brace directly on open wounds or infected areas.

- Make sure the brace does not come in direct contact with ointment or other chemical products.

### Application

A competent healthcare professional should perform the first fitting and product adjustments. Choose either AIR or Foam padding, what suits best the injury conditions.

- Apply the padding, with the black backside onto the hook strips, inside the shells. Ensure this padding is centered perfectly so the plastic shell edges are covered all around with padding.
- Put on a sock first in case no extreme pain or excessive swelling prevent you from doing this.
- Adjust the heel strap to the patient's heel width by re-positioning the vertical straps on both sides of the shells.
- Place the foot between the two plastic shells.
- Tighten the shells around the foot with the horizontal bottom strap until comfortable.
- Insert the foot with ankle brace in the shoe.
- Tighten the horizontal top strap to complete the fixation of the shells onto the ankle.
- For a proper fit, repeat steps 5 and 7 until you achieved optimal comfort.

### Care & Cleaning

Remove the foam padding and the heel closure straps from the splint. Close the hook of the straps. Hand-wash the straps and foam in lukewarm water (max 35°C) using a mild soap. Rinse with water and allow it to dry at room temperature. Do not iron. Clean the plastic shells with a damp cloth. Be sure that the materials are thoroughly dry before re-applying

### ⚠ Important information

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
- This brace should only be applied in accordance with the instructions mentioned in this manual.

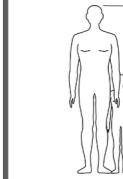
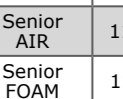
- The product is intended for single patient use.

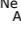
- No liability is accepted for incorrect product use.

- Guarantee is bound to statutory regulations.

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