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**EN** These products are only to be applied by medical specialists. In order to dispose of the product, comply with the legal norms of your country. If any serious incident that has occurred in relation to the device should be reported to manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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**EN** Please consult with your physician if any allergic symptom occurs. Do not use continuously for more than 24 hours. Daily check of skin conditions is recommended.

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**EN** Class I medical device, non sterile, no measurement function. This product fulfils the Medical Device Regulation (EU) 2017/745.

For more information please consult our product catalogue or visit [www.nordicare.se](http://www.nordicare.se).  
For further questions, contact your doctor, health practitioner or medical equivalent - or NordiCare: +46 42-35 22 20, [info@nordicare.se](mailto:info@nordicare.se)

**Zhangpu Ortholife Rehabilitation Co., Ltd.**  
No.6 Lanli West Road,  
Suian Town Zhangpu County,  
Zhangzhou City, Fujian Province,  
363200, China

**SUNGO Europe B.V.**  
Fascinatio Boulevard 522, Unit 1.7, 2909VA  
Capelle aan den IJssel, The Netherlands

**NordiCare Ortopedi och Rehab AB**  
Solrosvägen 1, SE-263 62 Viken, Sweden.  
Tel +46 42 35 22 20, Fax +46 42 35 22 21  
[info@nordicare.se](mailto:info@nordicare.se), [www.nordicare.se](http://www.nordicare.se)

**MD** Class I **CE**

This packaging can be recycled as paper and the bag as plastic.

**DISPOSAL OF THE PRODUCT**  
Do not dispose of the product in the nature. Dispose of the product in accordance with the waste laws of your municipality and/or country. The product does not present a risk of contamination.

# NordiCare®

## Measurements & Size

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**EN** Measure the foot length.

**NO** Mål fotlengden.

| Foot length (cm) | 22-25,5 | 26-28 | 28,5-30,5 | 31+ |
|------------------|---------|-------|-----------|-----|
| Size             | S       | M     | L         | XL  |

# Super Walking Boot

art no 80202, 80203

## WALKER

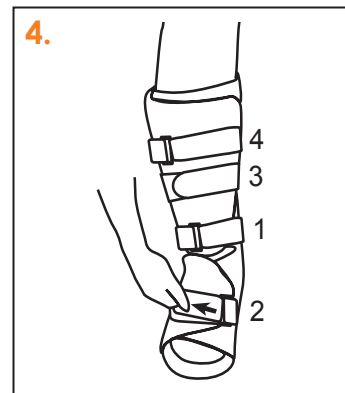
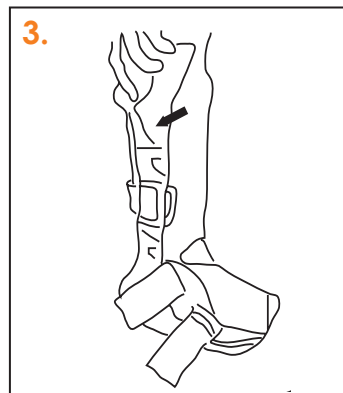
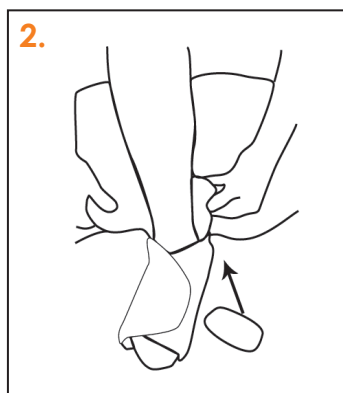
# INFORMATION FOR Super Walking Boot - WALKER

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**EN** 1. Place foot and leg inside foam liner and fasten it snugly.  
2. Additional pads can be placed inside the liner, or between the liner and the walker frame for better fitting and comfort.  
3. Pull uprights apart and place foot with liner into walker ensuring heel is against posterior border of walker. Uprights should be centered at the ankle joint. Apply pressure to ensure uprights are fixed to liner.  
4. Fasten ankle, foot and calf strap in order. Tighten straps until snug and comfortable.

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**EN** Stable foot or ankle fracture. Severe ankle sprain. Post surgery application.

**Contraindications:** Paraesthesia. Severe varicose vein. Skin disorders or injuries in the relevant area. Impaired lymphatic drainage.

## Material Composition

**Rails:** 90% Polypropylene (PP), 10% Polyamide (PA).

**Outsole:** 100% Thermoplastic Rubber (TPR).

**Padding:** 30% Polyester, 40% Polyurethane (PU), 20% Polyamide (PA), 7% EVA, 3% POM.

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**EN** Close all straps before washing. Hand wash in cold water with mild soap. Dry clean, bleach or tumble is prohibited. Natural dry, and do not expose this brace to any heat source.