# INFORMATION FOR Stratus - AIR WALKER

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**EN** These products are only to be applied by medical specialists. In order to dispose of the product, comply with the legal norms of your country. If any serious incident that has occurred in relation to the device should be reported to manufacturer and the competent authority of the Member State in which the user and/or patient is established.

## Material Composition

45% Polypropylene (PP),  
15% Thermoplastic Rubber (TPR),  
5% Thermoplastic Polyurethane (TPU),  
5% EVA, 10% Polyester, 5% Nylon,  
15% Polyurethane (PU).  
Latex-free.

For more information please consult our product catalogue or visit [www.nordicare.se](http://www.nordicare.se).  
For further questions, contact your doctor, health practitioner or medical equivalent - or NordiCare: +46 42-35 22 20, [info@nordicare.se](mailto:info@nordicare.se)

**Zhangpu Ortholife Rehabilitation Co., Ltd.**  
No.6 Lanli West Road,  
Suian Town Zhangpu County,  
Zhangzhou City, Fujian Province,  
363200, China

**SUNGO Europe B.V.**  
Fascinatio Boulevard 522, Unit 1.7, 2909VA  
Capelle aan den IJssel, The Netherlands

**NordiCare Ortopedi och Rehab AB**  
Solrosvägen 1, SE-263 62 Viken, Sweden.  
Tel +46 42 35 22 20, Fax +46 42 35 22 21  
[info@nordicare.se](mailto:info@nordicare.se), [www.nordicare.se](http://www.nordicare.se)

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**EN** Please consult with your physician if any allergic symptom occurs. Do not use continuously for more than 24 hours. Daily check of skin conditions is recommended.

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**EN** Class I medical device, non sterile, no measurement function. This product fulfils the Medical Device Regulation (EU) 2017/745.

**MD** Class I **CE**

This packaging can be recycled as paper and the bag as plastic.

**DISPOSAL OF THE PRODUCT**  
Do not dispose the product in the nature. Dispose of the product in accordance with the waste laws of your municipality and/or country. The product does not present a risk of contamination.

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**EN 1.** Place foot and leg inside foam liner and fasten it snugly.

**2.** Additional pads can be placed inside the liner, or between the liner and the walker frame for better fitting and comfort.

**3.** Place wrapped foot into Walker. Place front panel to ensure protection and support.

**4.** Fasten ankle, foot and calf strap in order. Tighten straps until snug and comfortable.

### How To Adjust Aircell Pressure:

**1.** Press the air pump to get inflation control.  
**2.** Press the valve for deflation.

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**EN** Stable foot or ankle fracture. Severe ankle sprain. Post surgery application. Achilles rupture combined with wedges.

**Contraindications:** Paraesthesia. Severe varicose vein. Skin disorders or injuries in the relevant area. Impaired lymphatic drainage.

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**EN** Provide support and comfort following trauma or post-operative procedure.

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**EN** Close all straps before washing. Hand wash in cold water with mild soap. Dry clean, bleach or tumble is prohibited. Natural dry, and do not expose this brace to any heat source.

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**EN** A heel wedge can be used as an accessory if an increased height is wanted for the plantarflection. Independent wedges are assembled together to the desired height. Toe-, hygiene- and rainprotection as well as socks is also possible to order.