Endodonti

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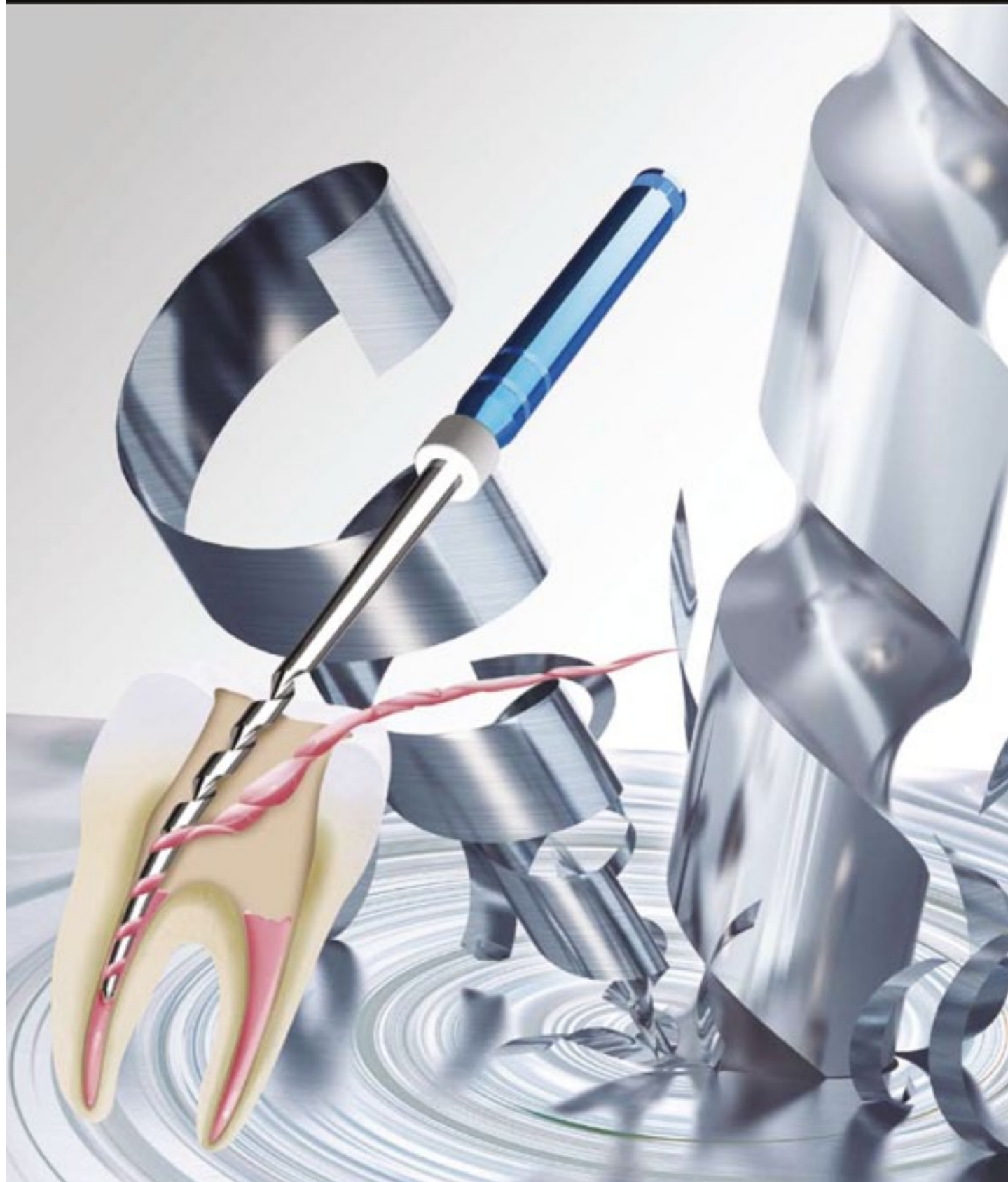
STEP 1

From crown to middle

- Select 1S/#70 or 2S/#50. (Depend on the shape of root canal)
- Attach the File on the top of GP (Gutta Percha) and rotate softly
- 1st groove catching the GP and file goes into the deep naturally.
- Pause the file 1-2 seconds at around the middle of the canal. (before the curve)
- Move the file in pull-stroke motion and remove the debris. STEP1-Finish.

STEP 2

From the middle to the apical 1~2mm

- Select 3N/#40 or 4N/#30. (Depend on curve of root canal)
- Set a rubber stopper length of 1-2mm upto the apex.
- Pause the file 1-2 seconds at around the middle~1-2mm short from the apex.
- Move the file in pull-stroke motion and remove the debris. STEP2-Finish.

Hyperelasticity+Shape Memory Effect
Take Note Thermally treated 5mm tip Shape Memory Effect files increase flexibility and fatigue resistance (compared with conventional products)

Examples

Provided by Dr. Yasuhisa Tsujimoto Nihon University School of Dentistry at Matsudo Department of Dentistry

FEATURES

- Stress-free and easy to use, with torque control and reversal !
- Fast & dynamic removal !
- No need for solvents !

Max. Allowable Speed 1000±500min⁻¹

Model Name	Size	Taper	Working Length	Material	Pack	Art. nr	Pris	
1S	#70	0.04	16mm	Stainless	4pcs	401022	59:-	
2S	#50	0.04	18mm	Stainless	4pcs	401023	59:-	
3N	#40	0.04	21mm	NiTi	4pcs	401024	119:-	
4N	#30	0.04	21mm	NiTi	4pcs	401025	119:-	
Assorted Pack	MÄSSPRIS! 87:- exkl. moms	#70×1 #50×1 #40×1 #30×1	0.04	16mm×1 18mm×1 21mm×1 21mm×1	Stainless 2pcs. NiTi 2pcs.	4pcs	401026	87:-

⚠ Please read attached instructions carefully before use

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NYHET!

Silk™ by MANI®

**Predictable Results + Silky-Smooth Feel
A Winning Combination**

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Dr's Finder

Nu lanserar vi den senaste teknologin inom Apex Lokalisering.

Dr's Finder provide accurate with newest technology.
Dr's Finder is the best solution for endodontic treatment.

4,399 sek

Included accessories

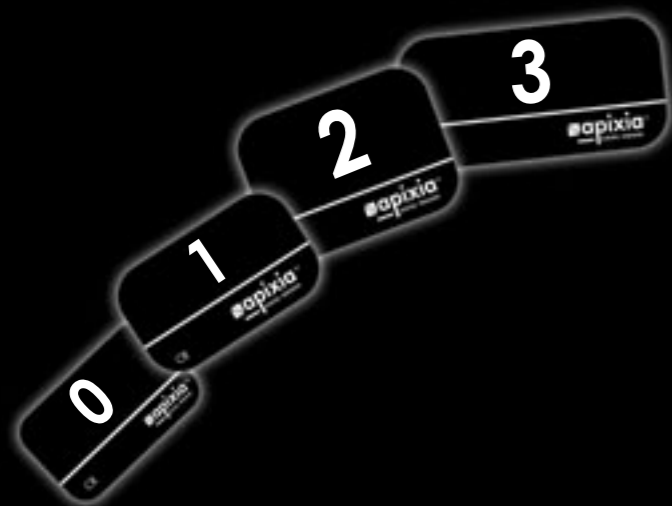
Smart Apex Locator

www.normmedico.se
order@normmedico.com
Tel +46 (0) 8 751 7777

NORMEDICO PRESENTERAR
DIGIREX® PSP SCANNER
 DIGITAL DENTAL RADIOGRAPHY IMAGING SYSTEM

DIGIREX® PSP
 PHOTO-STIMULABLE PHOSPHOR PLATE

Produkt	Art. nr
Apixia Scanner	902780
Phosphor Plate size 0, sets: 4	902782
Phosphor Plate size 2, sets: 5	902783
Phosphor Plate size 3, sets: 2	902784
Endomax Hanppiece Endodontic, 5 sets	902785
E barrier envelope #0, 2 box	902786
Phosphor Plate Barrier, 2 box, Envelope (for size1&2)	902787
Plate barrier envelope #3, 2 box	902788

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End Cutting

FIG NO.	840
DIAMETER	012
INTERNATIONAL REF.	EX-18
HEAD LENGTH (MM)	END CUT
COARSE	
MEDIUM	
FINE	
EXTRA FINE	

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