



RI.MOS. s.r.l. viale Gramsci, 29 – 41037 MIRANDOLA (MO) ITALY – tel. +39 0535 25755
fax +39 0535 25509 - e-mail: info@rimos.com – www.rimos.com

GINRAM®
PRODOTTI MONOUSO STERILI PER LA GINECOLOGIA
 Gynaecological disposable sterile devices


TIRAPALLE
Pinza di Pozzi-Palmer
Pozzi-Palmer forceps

COD.720131

SE Bruksanvisning för Palmer pincettsax

Ta ut produkten. Introducera Tirapalle i vaginalkanalen, öppna hakarna och skjut upp den till livmoderhalsen. Ta tag i livmoderhalsen och låt hakarna penetrera vävnaden. Lås centralklämman och den inbuktande klämman innan du drar i livmoderhalsen.

Varning: Läs instruktionerna före användning, Återanvänd inte. Använd inte om förpackningen är trasig. Används omedelbart efter öppning. Släng efter användning enligt lokala föreskrifter. Skall endast användas av medicinskt tränad personal.

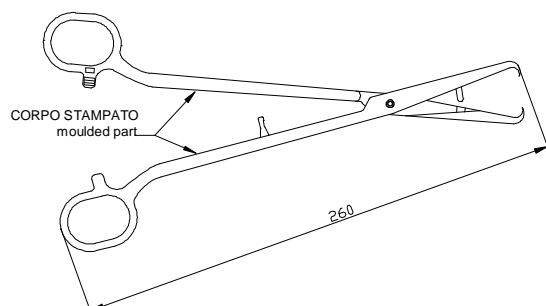

GB Instructions for use

Expose the portio. Introduce the forceps in the vaginal canal, then open the jaw and draw it up to the uterus neck. Grasp the neck firmly and let the jaws penetrate into the tissues. Lock the central clamp and the indented clamp before starting pulling the uterus neck.

Caution: Read the instructions before use. Do not reuse. Use only if the package is not damaged. Use immediately after opening. Dispose of after use according to the laws in force. For use only under trained medical staff